
Edwards

Pressure Controller Kit • Kit de surveillance de pression • Druckregler-Set • Kit del controlador de presión • Kit pressostato • Drukregelingsuitrusting • Trykstyringssæt • Tryckstyrenhetssats • Kit ρυθμιστή πίεσης • Kit de controlador de pressão • Souprava regulátoru tlaku • Nyomákszabályozó készlet • Zestaw modułu ciśnieniowego • Súprava regulátora tlaku • Trykkontrollersett • Paineensäädinsarja • Кутія за регулятора на налягането • Set regulator presiune • Rõhuregulaatorikomplekt • Slėgio valdymo įtaiso rinkinys • Spiediena kontrollera komplekts • Basınç Kontrolör Kiti • Комплект регулятора давления • Komplet kontrolera pritiska • 压力控制器套件 • 압력 제어장치 키트 • 圧制御装置キット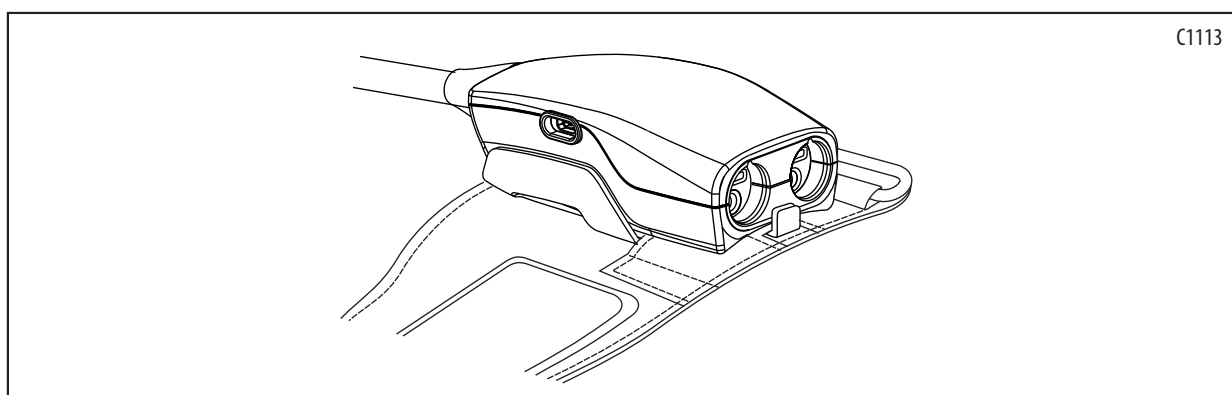

DIRECTORY

English.....	2	Português.....	20	Eesti.....	38
Français.....	4	Česky.....	22	Lietuvių.....	40
Deutsch.....	6	Magyar.....	24	Latviešu.....	42
Español.....	8	Polski.....	26	Türkçe.....	44
Italiano.....	10	Slovensky.....	28	Русский.....	46
Nederlands.....	12	Norsk.....	30	Srpski.....	48
Dansk.....	14	Suomi.....	32	中文.....	50
Svenska.....	16	Български.....	34	한국어.....	52
Ελληνικά.....	18	Română.....	36	日本語.....	54

Pressure Controller Kit

REF PC2K

For Figures 1 through 4, please refer to page 56.

Description

The pressure controller kit consists of a pressure controller (PC2) and accompanying band (PC2B). The pressure controller can be used with the EV1000 clinical platform NI and HemoSphere advanced monitoring platform to noninvasively measure blood pressure and key hemodynamic parameters using a compatible Edwards finger cuff wrapped around the finger. The pressure controller is worn on the patient's wrist and connects to the compatible Edwards noninvasive monitoring system, heart reference sensor and compatible Edwards finger cuff(s) (Fig 1).

Intended Use

The pressure controller is intended for use with an Edwards compatible noninvasive monitoring system - composed of compatible monitor, pressure source (pump), heart reference sensor, compatible Edwards finger cuff(s) and pressure controller - for noninvasive measurement of blood pressure and associated hemodynamic parameters. Refer to the operator's manual of the compatible Edwards monitor being used for specific information on the intended use environment and patient population.

Instructions for Use

Apply the Pressure Controller

1. Wrap the pressure controller band around the patient's wrist (Fig 2, step 1).
2. Snap the pressure controller into the plastic sleeve of the band, making sure that the cuff connectors are facing towards the fingers (Fig 2, step 2).
3. Attach the pressure controller cable to the port on compatible Edwards noninvasive monitoring system (Fig 1A).
4. Remove the plastic connector caps in order to connect the finger cuff and heart reference sensor to the pressure controller according to the Instructions for Use supplied with the particular units (Fig 3, step 1-2), or consult the operator's manual of the EV1000 clinical platform NI and HemoSphere advanced monitoring platform.
Note: It is recommended that the cuff connector caps be kept and used to protect the pressure controller against the ingress of water and dirt when only a single cuff is used (Fig 3, step 3).
5. Once the heart reference sensor (HRS) is connected to the pressure controller, then place the pressure controller cover (PCCVR) on the pressure controller.

Edwards, Edwards Lifesciences, the stylized E logo, EV1000, and HemoSphere are trademarks of Edwards Lifesciences Corporation. All other trademarks are property of their respective owners.

WARNING: Do not overtighten the pressure controller band.

WARNING: Do not apply pressure controller band on injured skin as this can cause further injury.

Remove the Pressure Controller Band

To remove the pressure controller from the pressure controller band:

1. Pull the sleeve slightly outwards (Fig 4, step 1) and tilt the pressure controller to remove it from the sleeve (Fig 4, step 2).

Cleaning Instructions

The pressure controller can be cleaned using the following disinfectants:

- 70% isopropyl alcohol solution
 - 10% sodium hypochlorite water solution
1. Moisten a clean cloth with disinfectant and wipe the surfaces.
 2. Dry the surface with a clean dry cloth.

CAUTION: Do not immerse the pressure controller.

CAUTION: Do not disinfect the pressure controller by autoclave or gas sterilization.

CAUTION: Do not immerse any cable connectors in fluid.

The pressure controller band is intended for limited reuse. The operator shall assess whether reuse is appropriate. When reused, the same cleaning instructions can be followed as specified for the pressure controller.

Accessories

Model Number	Accessory Description
PC2K	Pressure Controller Kit
PC2B	Pressure Controller Band (Multi Pack of 5)
PC2CCC	Pressure Controller Cuff Connector Caps (Multi Pack of 25)
PCCVR	Pressure Controller Cover

Warranty

The pressure controller is warranted for a period of one (1) year from the date of purchase when used in accordance with the directions for use. This warranty does not include the pressure controller band. For a full description of the warranty conditions refer to the operator's manual of the EV1000 clinical platform NI and HemoSphere advanced monitoring platform. Both the pressure controller and the pressure controller band are non-serviceable parts.

Technical Assistance

For technical assistance, please call Edwards Technical Support at the following telephone numbers:

Inside the U.S. and Canada

(24 hours): 800.822.9837

Outside the U.S. and Canada

(24 hours): 949.250.2222


In the UK: 0870 606 2040 - Option 4

In Ireland: 01 8211012 Option 4

This document is subject to change without prior notice. Refer to the latest version of the EV1000 clinical platform NI and HemoSphere advanced monitoring platform operator's manual for additional information.

Users and/or patients should report any serious incidents to the manufacturer and the Competent Authority of the Member State in which the user and/or patient is established.

Symbol Legend

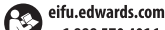Symbols	Description
	Catalogue Number
	Separate collection for electrical and electronic equipment in accordance with EC Directive 2012/19/EU
	Medical Device
	Serial number
	LOT number
	Quantity
	Conformité Européenne (CE Mark)
	Authorized representative in the European Community
	Date of Manufacture
	Manufacturer
	Follow instructions for use
	Follow instructions for use on the website
	Caution
	MR Unsafe
Rx only	Caution: Federal (USA) law restricts this device to sale by or on the order of a physician
	No serviceable parts inside
	Keep dry
	Fragile, handle with care
	Keep away from sunlight
IP4X	Extent of protection against the ingress of objects
	Temperature limit
	Humidity limitation
	Atmospheric pressure limitation
Note: Not all symbols may be included in the labeling of this product.	

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© Copyright 2019, Edwards Lifesciences LLC
All rights reserved.
Edwards Lifesciences Services GmbH
Edwards Lifesciences LLC