Edwards Lifesciences

DIRECTORY

English	1	Português.....	28	Eesti.....	55
Français.....	4	Česky.....	31	Lietuvių.....	58
Deutsch.....	7	Magyar	34	Latviešu.....	61
Español	10	Polski.....	37	Türkçe	64
Italiano	13	Slovensky	40	Русский.....	67
Nederlands.....	16	Norsk	43	Srpski.....	70
Dansk.....	19	Suomi	46	简体中文	73
Svenska.....	22	Български	49	한국어	76
Ελληνικά	25	Română.....	52	日本語	79

English

ClearSight Finger Cuff

REF CSCS, CSCM, CSCL

CC1091

ClearSight Finger Cuff

Description

The ClearSight finger cuff can be used with the EV1000 clinical platform NI or HemoSphere advanced monitor to noninvasively measure blood pressure and key hemodynamic

parameters. The ClearSight finger cuff wraps around the finger.

Indications

The ClearSight finger cuff is indicated for patients over 18 years of age to noninvasively measure blood pressure and associated hemodynamic parameters when used with EV1000 clinical platform NI or HemoSphere advanced monitoring platform.

Edwards, Edwards Lifesciences, the stylized E logo, ClearSight, EV1000, and HemoSphere are trademarks of Edwards Lifesciences Corporation. All other trademarks are the property of their respective owners.

Dual Cuff application

The EV1000 clinical platform NI and HemoSphere advanced monitor offer the possibility to connect two finger cuffs and alternate the measurement between two fingers. It is recommended to apply an additional finger cuff to a second finger on the same hand when continuous monitoring is expected to last longer than 8 hours, or to increase patient comfort. Continuous measurement on one finger is limited to 8 hours.

Single Patient Use

The ClearSight finger cuff is designed for single patient use. Upon starting a measurement, the finger cuff can be used and re-applied for up to 72 hours on one patient.

WARNINGS:

Do not apply the ClearSight finger cuff on injured skin as this may cause further injury.

To reduce the risk of skin irritation and tissue damage, do not monitor longer than 8 hours continuously on a single finger. To continue to monitor beyond 8 hours, apply the ClearSight finger cuff to another finger or use two cuffs to measure.

Do not use two ClearSight finger cuffs simultaneously on the same finger.

Measurement on one finger in contradiction with the instructions for use may affect patient comfort and/or lead to minor injuries.

Do not apply ClearSight finger cuff(s) on a hand/finger when external constriction (that may prevent circulation to the hand/finger) is present.

Do not apply ClearSight finger cuff(s) on a hand/finger when a second blood pressure measurement device is actively monitoring on the same arm (or hand/finger).

Do not overtighten the ClearSight finger cuff(s).

Do not use the finger cuff with magnetic resonance imaging.

Improper ClearSight finger cuff placement or sizing can lead to inaccurate monitoring.

CAUTIONS:

Do not use a damaged ClearSight finger cuff. This may result in inaccurate measurements or may damage the EV1000 clinical platform NI or HemoSphere advanced monitor.

Never bend a ClearSight finger cuff to a flat shape, it will damage the finger cuff and affect measurement accuracy.

Excessive ambient light may interfere with ClearSight finger cuff measurements.

Always disconnect the ClearSight finger cuff when it is not wrapped around a finger, to prevent damage by accidental over-inflation.

The effectiveness of the finger cuffs have not been established in pre-eclamptic patients.

Instructions for Use

Refer to pages 82 to 83 of this instructions for use to show figures corresponding to the instructions.

Select finger

1. Select the middle, ring or index finger.
Note: Do not apply the finger cuff on the thumb or previously fractured fingers.

Define Finger Cuff size

2. Take a ClearSight finger cuff sizing aid and wrap the sizing aid around the middle phalanx of the finger by pulling the color coded smaller end through the slot to create a snug fit (Fig 1).
3. The black arrow indicates the suitable finger cuff size (Fig 2). Match the indicated color with the correct finger cuff size.

Apply Finger Cuff

4. Open the finger cuff and place the finger between the lines, with the finger cuff centered between the knuckles (Fig 3).
5. Gently lead the cuff cable in between two fingers to the other side (back) of the hand (Fig 3).

6. Tightly wrap the cuff around the finger and ensure that the correct size cuff has been chosen by checking that the outer edge lines up within the green area of the cuff when wrapped tightly (Fig 4).
NOTE: Do not rotate the finger cuff after application (Fig 5).
7. Connect the finger cuff to the pressure controller (Fig 6).

Apply second Finger Cuff

It is recommended to apply an additional finger cuff to a second finger on the same hand when continuous monitoring is expected to last longer than 8 hours, or to increase patient comfort.

8. Repeat Steps 1-7 to apply the second finger cuff.

NOTE: When only a single finger cuff is used, it is recommended that the second cuff connector opening in the pressure controller is closed with the appropriate cuff connector cap (PC2CCC) to prevent ingress by liquid and dirt (Fig 6).

Cleaning instructions

The ClearSight finger cuff is designed for single patient use only. Do not attempt to clean and reuse the ClearSight finger cuff on more than a single patient.

Accessories

Model Number	Description
CSCS	ClearSight Finger Cuff Small (Multi Pack of 5)
CSCM	ClearSight Finger Cuff Medium (Multi Pack of 5)
CSCL	ClearSight Finger Cuff Large (Multi Pack of 5)
PC2CCC	Pressure Controller Cuff Connector Caps (Multi Pack of 25)

Warranty

The ClearSight finger cuff is for single patient use only. The ClearSight finger cuffs are warranted at time of delivery to the end user only. The ClearSight finger cuff is a non-serviceable part.

Technical Assistance

For technical assistance, please call Edwards Technical Support at the following telephone numbers:

Inside the U.S. and Canada
(24 hours): 800.822.9837
Outside the U.S. and Canada
(24 hours): 949.250.2222
In the UK: 0870 606 2040 - Option 4
In Ireland: 01 8211012 Option 4

This document is subject to change without prior notice. Refer to the latest version of the EV1000 clinical platform NI or HemoSphere advanced monitoring platform NI operator’s Manual for more additional information.

Users and/or patients should report any serious incidents to the manufacturer and the Competent Authority of the Member State in which the user and/or patient is established.

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© Copyright 2020, Edwards Lifesciences LLC
All rights reserved.
Edwards Lifesciences Services GmbH
Edwards Lifesciences AG
Edwards Lifesciences LLC