Edwards

DIRECTORY

English.....	1	Slovensky	14
Français.....	2	Norsk	15
Deutsch.....	3	Suomi	16
Español	4	Български	17
Italiano	5	Română	18
Nederlands.....	6	Eesti.....	19
Dansk.....	7	Lietuvių.....	20
Svenska	8	Latviešu	21
Ελληνικά	9	Türkçe	22
Português.....	10	Русский.....	23
Česky	11	日本語.....	24
Magyar	12	中文.....	25
Polski.....	13		

English

Roll Stand

REF EVRS

For figures 1 through 7, refer to pages 21 through 22.

Intended Use

The roll stand is intended to be used with the EV1000 clinical platform. See EV1000 clinical platform operator's manual for more details.

Installation Note:

Separate installation kits and accessories may be included with the roll stand. Read all installation guides before assembling the roll stand or mounting instruments or accessories.

Instructions for Use

Attaching Roll Stand Post to the Base

1. Insert the post in the base and lay assembly on its side for access to bottom of the base (See page 21, Figure 1).
2. Using a 1/2" [13 mm] wrench, attach the post to the base with one (1) 5/16-18 x 1" HHCS, 5/16" split lock washer, and 5/16" flat washer.

Attaching the Basket to the Post

1. Install both nylon clips at an angle, in the center of the basket. Rotate clips until both are horizontal (See page 21, Figure 2).

2. Slide the clips to the top and bottom of basket. The top clip should be touching the top horizontal wire, and the bottom clip should be touching the lower reinforcing wire (See page 21, Figure 3).
3. Slide the clips over the top of the post (See page 21, Figure 4). Position the basket at the desired height. Tighten the clips using the supplied #8-32 machine screws and hex nuts (See page 22, Figure 5).

Attaching Top Plate

1. Connect top plate to the top of the pole using the three screws provided (See page 22, Figure 6).

Maintenance

All fasteners associated with the mounting system should be inspected periodically and tightened as necessary. Roll stand may be cleaned with any non-abrasive solution as approved by the facility in which the product will be used.

Warnings

- DO NOT use this product without first reading and understanding the instructions contained in this booklet. If you are unable to understand the Warnings, Caution or Instructions, contact a healthcare professional, dealer or technical personnel before use — otherwise serious bodily injury or product damage may occur.
- All wheels must be in contact with the floor at ALL TIMES.
- DO NOT use the roll stand while walking backwards down gradients or climbing stairs, curbs or to go over obstacles. Serious risk of fall or injury may occur.

Edwards, Edwards Lifesciences, the stylized E logo, and EV1000 are trademarks of Edwards Lifesciences Corporation. All other trademarks are the property of their respective owners.

- Engage all wheel locks when roll stand is not in motion.
- Use care when moving roll stand with a patient. Avoid wet or slick surfaces, wires, cables, or tubes and any uneven surfaces that may cause roll stand to tip or fall.
- Only use accessories and spare parts authorized by GCX Corporation.
- DO NOT attach roll stand to other objects.
- DO NOT use in MRI areas.
- DO NOT use the roll stand for support or as a walking aid. Do not attempt to support your weight or use as a stand assist.

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Technical Assistance

For technical assistance, please call Edwards Technical Support at the following telephone numbers:

Inside the U.S. and Canada (24 hours): 800.822.9837
Outside the U.S. and Canada (24 hours): 949.250.2222
In the UK: 0870 606 2040 - Option 4
In Ireland: 01 8211012 Option 4

CAUTION: Federal (USA) law restricts this device to sale by or on the order of a physician.

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© Copyright 2020, Edwards Lifesciences LLC
All rights reserved.
Edwards Lifesciences Services GmbH
Edwards Lifesciences LLC