



## Viewfinder Remote App Release Notes

The following is a list of known device implementation anomalies and troubleshooting steps associated with the listed software release version. The software version is listed on the About screen (Settings → System Options → About). These issues are continually updated and compiled as a result of ongoing product improvements.

### Viewfinder remote app anomalies for software release version 1.6.6

Known anomaly	Cause	Suggested actions
When using HemoSphere monitor version 02.03.XXX.XXX or prior, the technology type label on the patient monitoring session does not differentiate between "Acumen IQ sensor" and "FloTrac sensor" or between "Acumen IQ cuff" and "ClearSight cuff" properly.	HemoSphere advanced monitor does not send the displayed technology type to Viewfinder hub for display on Viewfinder remote app.	Refer to HemoSphere advanced monitor to view correct technology type.



06/23  
10050052005 A / DOC-0222669 A  
© Copyright 2023, Edwards Lifesciences LLC  
All rights reserved



**Edwards Lifesciences LLC**  
One Edwards Way  
Irvine, CA 92614 USA  
Made in USA

Telephone 949.250.2500  
800.424.3278  
FAX 949.250.2525

Web IFU