Directory

English (en).....	1
Français (fr).....	2
Deutsch (de).....	3
Español (es).....	4
Italiano (it).....	5
Nederlands (nl).....	6
Dansk (da).....	7
Svenska (sv).....	8
Ελληνικά (el).....	9
Português (pt).....	10
Česky (cs).....	11
Magyar (hu).....	12
Polski (pl).....	13
Slovensky (sk).....	14
Norsk (no).....	15
Suomi (fi).....	16
Български (bg).....	17
Română (ro).....	18
Eesti (et).....	19
Lietuvių (lt).....	20
Latviešu (lv).....	21
Türkçe (tr).....	22
Русский (ru).....	23
Srpski (sr).....	24
Hrvatski (hr).....	25

English

HemoSphere Vita Monitor Release Notes

The following is a list of known device implementation anomalies and troubleshooting steps associated with the listed software release version. The software version is listed on the Versions screen (Settings → Help → Versions). These issues are continually updated and compiled as a result of ongoing product improvements.

HemoSphere Vita monitor anomalies for software release version 03.02.000.006

Known anomaly	Cause	Suggested actions
The HRS Calibration screen instructions are not displayed during active monitoring after the VitaWave module is disconnected and then reconnected again.	HRS calibration screen is not refreshing when VitaWave module is disconnected and then reconnected.	Exit out of the VitaWave monitor settings screen and then re-enter the same screen.
During monitoring, a fatal error occurs and the monitor displays the following message: "System.OutOfMemoryException".	System exceeds the expected memory allocation.	Power cycle the monitor.
When monitoring using VitaWave module with software version 01.01.004, the generic fault message "Fault: HemoSphere Vita Module" is displayed instead of "Fault: L-Tech Module Slot - Incompatible Software Version".	VitaWave module firmware versions prior to 01.03.xxx do not support the VitaWave Cuff.	Contact Edwards Technical Support for the latest firmware.
Key parameters configured on the monitoring screen are not saved after monitor is powered off and on again.	The monitor is shut down before key parameter display settings are saved.	Reconfigure desired parameters on screen upon monitor reboot.

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© Copyright 2024, Edwards Lifesciences LLC
All rights reserved
Edwards Lifesciences LLC