



Edwards

HemoSphere Advanced Monitor Release Notes

The following is a list of known device implementation anomalies and troubleshooting steps associated with the listed software release version. The software version is listed on the Versions screen (Settings → Help → Versions). These issues are continually updated and compiled as a result of ongoing product improvements.

HemoSphere advanced monitor anomalies for software release version 04.05.000.001

Known anomaly	Cause	Suggested actions
If the clinical data download is unsuccessful, the Home button becomes disabled.	The cause for this anomaly is still under investigation.	Remove the USB from the HemoSphere advanced monitor. The Home button is now enabled again.
When selecting a new Parameter, the words "Pressure" and "Oximetry" are not translated.	The translations were not implemented correctly.	The translated parameter names remain visible to the user; only the technology description is affected by this issue.



03/26
10061042001 B / DOC-0245620 B
© Copyright 2026, Edwards Lifesciences LLC
All rights reserved

 **Edwards Lifesciences LLC**
One Edwards Way
Irvine, CA 92614 USA
Made in USA

Telephone 949.250.2500
800.424.3278
FAX 949.250.2525

Web IFU