



Edwards

HemoSphere Vita Monitor Release Notes

The following is a list of known device implementation anomalies and troubleshooting steps associated with the listed software release version. The software version is listed on the Versions screen (Settings → Help → Versions). These issues are continually updated and compiled as a result of ongoing product improvements.

HemoSphere Vita monitor anomalies for software release version 04.05.000.000

Known anomaly	Cause	Suggested actions
If the clinical data download is unsuccessful, the Home button becomes disabled.	The cause for this anomaly is still under investigation.	Remove the USB from the HemoSphere Vita monitor. The Home button is now enabled again.



09/25
10061046001 A / DOC-0245631 A
© Copyright 2025, Edwards Lifesciences LLC
All rights reserved



Edwards Lifesciences LLC
One Edwards Way
Irvine, CA 92614 USA
Made in USA

Telephone 949.250.2500
800.424.3278
FAX 949.250.2525

Web IFU