Edwards

Directory

English (en).....	2
Français (fr).....	7
Deutsch (de).....	13
Español (es).....	19
Italiano (it).....	25
Nederlands (nl).....	31
Dansk (da).....	37
Svenska (sv).....	43
Ελληνικά (el).....	48
Português (pt).....	54
Česky (cs).....	60
Magyar (hu).....	66
Polski (pl).....	72
Slovensky (sk).....	78
Norsk (no).....	83
Suomi (fi).....	88
Български (bg).....	93
Română (ro).....	99
Eesti (et).....	105
Lietuvių (lt).....	110
Latviešu (lv).....	116

Türkçe (tr).....	121
Русский (ru).....	126
Srpski (sr).....	132
Hrvatski (hr).....	138
Figures ■ Figures ■ Abbildungen	
■ Figuras ■ Figure ■ Afbeeldingen	
■ Figurer ■ Figurer ■ Εικόνες ■ Figuras	
■ Obrázky ■ Ábrák ■ Rysunki	
■ Obrázky ■ Figurer ■ Kuvat ■ Фигури	
■ Figuri ■ Joonised ■ Paveikslai	
■ Attēli ■ Şekiller ■ Рисунки ■ Slike	
■ Slike.....	144
Symbol Legend ■ Légende des symboles ■ Zeichenerklärung	
■ Significado de los símbolos	
■ Legenda dei simboli ■ Lijst met symbolen ■ Symbolforklaring	
■ Symbolförklaring ■ Υπόμνημα συμβόλων ■ Legenda dos símbolos	
■ Legenda se symboly	
■ Jelmagyarázat ■ Legenda symboli	
■ Vysvetlivky k symbolom	
■ Symbolforklaring ■ Merkkien selitykset ■ Легенда на символите	
■ Legendă de simboluri ■ Sümbolite seletus ■ Simbolių paaiškinimas	
■ Simbolu skaidrojums ■ Sembol Açıklamaları ■ Условные обозначения	
■ Legenda sa simbolima ■ Legenda simbola.....	147

English

ClearSight Finger Cuff

The devices described herein may not all be licensed in accordance with Canadian law or approved for sale in your specific region.

CSCS, CSCM, CSCL

CC1091

ClearSight Finger Cuff

Carefully read these instructions for use, which address the warnings, precautions, and residual risks for this medical device.

1.0 Description

The ClearSight finger cuffs, when used with an appropriate Edwards monitoring system, provide continuous, noninvasive hemodynamic monitoring and the applicable derived parameters. The ClearSight finger cuffs utilize the volume-clamp method to measure blood pressure with an inflatable bladder wrapped around the middle phalanx of the finger. This device is supplied non-sterile. Potential risks include inappropriate/unintended treatment, adverse reaction to device materials, patient or clinician burns or electrical shock, peripheral ischemia, and/or minor tissue damage.

Edwards, Edwards Lifesciences, the stylized E logo are trademarks of Edwards Lifesciences Corporation. ClearSight, EV1000 and HemoSphere are trademarks of Becton, Dickinson and Company. All other trademarks are the property of their respective owners.

Device performance, including functional characteristics, have been verified in a comprehensive series of testing to support the safety and performance of the device for its intended use when used in accordance with the established Instructions For Use.

Device is used by clinicians who have been trained in safe use of noninvasive hemodynamic technologies in accordance with their institutional guidelines.

2.0 Intended Use/Purpose

The intended purpose is to noninvasively measure blood pressure and use the information to derive hemodynamic parameters when connected to the EV1000 clinical platform or HemoSphere advanced monitoring platform.

3.0 Indications

The ClearSight finger cuff is indicated for patients over 18 years of age to noninvasively measure blood pressure and associated hemodynamic parameters when used with EV1000 clinical platform NI or HemoSphere advanced monitoring platform.

4.0 Contraindications

In some patients with extreme contraction of the smooth muscle in the arteries and arterioles in the lower arm and hand, such as may be present in patients with Raynaud's disease, blood pressure measurement can become impossible.

5.0 Single Patient Use

The ClearSight finger cuff is designed for single patient use. Upon starting measurement, the finger cuff can be used and re-applied for up to 72 hours on one patient. After 8 hours of continuous monitoring on a single finger, the finger cuff should be re-applied to another finger.

6.0 WARNINGS:

- **Do not apply the ClearSight finger cuff on injured skin as this may cause further injury.**
- **To reduce the risk of skin irritation and tissue damage, do not monitor longer than 8 hours continuously on a single finger. To continue to monitor beyond 8 hours, apply the ClearSight finger cuff to another finger or use two cuffs to measure.**
- **Do not use two ClearSight finger cuffs simultaneously on the same finger.**
- **Measurement on one finger in contradiction with the instructions for use may affect patient comfort and/or lead to minor injuries.**
- **Do not apply ClearSight finger cuff(s) on a hand/finger when external constriction (that may prevent circulation to the hand/finger) is present.**
- **Do not apply ClearSight finger cuff(s) on a hand/finger when a second blood pressure measurement device is actively monitoring on the same arm (or hand/finger).**
- **Do not overtighten the ClearSight finger cuff(s).**
- **Do not use the finger cuff with magnetic resonance imaging.**
- **Improper ClearSight finger cuff placement or sizing can lead to inaccurate monitoring.**

7.0 CAUTIONS:

- **Do not use a damaged ClearSight finger cuff. This may result in inaccurate measurements or may damage the HemoSphere advanced monitor.**
- **Never bend a ClearSight finger cuff to a flat shape, it will damage the finger cuff and affect measurement accuracy.**
- **Excessive ambient light may interfere with ClearSight finger cuff measurements.**
- **Always disconnect the ClearSight finger cuff when it is not wrapped around a finger, to prevent damage by accidental overinflation.**
- **The effectiveness of the finger cuffs have not been established in pre-eclamptic patients.**

8.0 Instructions for Use

Refer to Figure 1 on page 144 to Figure 6 on page 146 of this instructions for use to show figures corresponding to the instructions.

8.1 Select finger

Step	Procedure
1	Select the middle, ring or index finger. Note: Do not apply the finger cuff on the thumb or previously fractured fingers. Note: Measure finger size at the finger location where the cuff will be applied to ensure the finger size is within the cuff size range.

8.2 Define Finger Cuff size

Step	Procedure
1	Take a ClearSight finger cuff sizing aid and wrap the sizing aid around the middle phalanx of the finger by pulling the color coded smaller end through the slot to create a snug fit Figure 1 on page 144.
2	The black arrow indicates the suitable finger cuff size Figure 2 on page 144. Match the indicated color with the correct finger cuff size.

8.3 Apply Finger Cuff

Step	Procedure
1	Open the finger cuff and place the finger between the lines, with the finger cuff centered between the knuckles (Figure 3 on page 145).
2	Gently lead the cuff cable in between two fingers to the other side (back) of the hand (Figure 3 on page 145).
3	Tightly wrap the cuff around the finger. After start of the measurement, ensure to obtain a good signal for accurate readings. (Figure 4 on page 145). Note: If a good signal is not obtained or quality of signal is not good, remove cuff and reposition on the finger Note: Do not rotate the Finger Cuff after application (Figure 5 on page 146).
4	Connect the finger cuff to the pressure controller (Figure 6 on page 146).

8.4 Apply second Finger Cuff

Step	Procedure
1	It is recommended to apply an additional finger cuff to a second finger on the same hand when continuous monitoring is expected to last longer than 8 hours, or to increase patient comfort. Continuous measurement on one finger is limited to 8 hours.
2	Repeat steps in sections Select finger , and Apply Finger Cuff to apply the second finger cuff. Note: When only a single finger cuff is used, it is recommended that the second cuff connector opening in the pressure controller is closed with the appropriate cuff connector cap (PC2CCC) to prevent ingress by liquid and dirt (Figure 6 on page 146).

9.0 Cleaning instructions

The ClearSight finger cuff is designed for single patient use only. Do not attempt to clean and reuse the ClearSight finger cuff on more than a single patient, such action could lead to illness or an adverse event as the device may not function as originally intended.

10.0 Recommendations

The Edwards noninvasive monitoring system will pause blood pressure measurements periodically per the monitoring system configuration settings. At this time, the fingertip should

be checked for perfusion and potential tissue damage. The fingertip should also be checked periodically according to hospital protocol.

11.0 MRI Safety

This device is MR-unsafe and poses hazards in the MRI environment. This device contains metallic components, which can experience RF-induced heating in the MRI environment.

12.0 Storage

Store in a cool, dry place.

13.0 Shelf Life

The shelf life is as marked on each package. Storage or usage beyond the expiration date may result in product deterioration and could lead to illness or an adverse event as the device may not function as originally intended.

14.0 Disposal

After patient contact, treat the device as biohazardous waste. Dispose of in accordance with hospital policy and local regulations.

15.0 Accessories

Model Number	Description
CSCS	ClearSight Finger Cuff Small (Multi Pack of 5)
CSCM	ClearSight Finger Cuff Medium (Multi Pack of 5)

Model Number	Description
CSCL	ClearSight Finger Cuff Large (Multi Pack of 5)
PC2C CC	Pressure Controller Cuff Connector Caps (Multi Pack of 25)

Users and/or patients should report any serious incidents to the manufacturer and the Competent Authority of the Member State in which the user and/or patient is established.

Refer to the symbol legend at the end of this document.

16.0 Warranty

The ClearSight finger cuff is for single patient use only. The ClearSight finger cuffs are warranted at time of delivery to the end user only. The ClearSight finger cuff is a non-serviceable part.

17.0 Technical Assistance

For technical assistance, please call Edwards Technical Support at the following telephone numbers:

Inside the U.S. and Canada

(24 hours): . . 800.822.9837

Outside the U.S. and Canada

(24 hours): . . 949.250.2222

In the UK: . . . 0870 606 2040 - Option 4

In Ireland: . . . 01 8211012 - Option 4

This document is subject to change without prior notice. Refer to the latest version of the EV1000 clinical platform NI or HemoSphere advanced monitoring platform NI operator's manual for more information.

CAUTION: Federal (USA) Law restricts this device to sale by or on the order of a physician.

Refer to the latest version of the Edwards noninvasive monitoring system operator's manual for more information.

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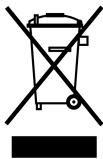
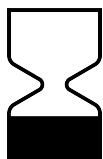
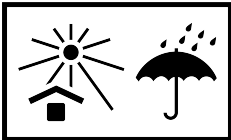
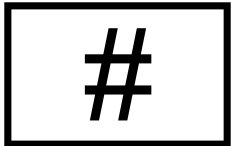
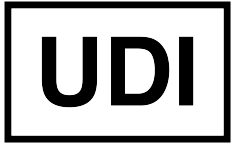
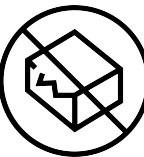
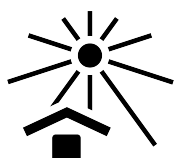
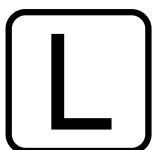
	English	Français	Deutsch	Español	Italiano
Rx only	Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.	Avertisse- ment : Federal (USA) law restricts this device to sale by or on the order of a physician.	Vorsicht: Federal (USA) law restricts this device to sale by or on the order of a physician.	Aviso: Federal (USA) law restricts this device to sale by or on the order of a physician.	Attenzione: Federal (USA) law restricts this device to sale by or on the order of a physician.
	Conformité Européenne (CE Mark)	Conformité européenne (marquage CE)	Conformité Européenne (CE- Kennzeichnung)	Conformité Européenne (Marca CE)	Conformité Européenne (Marchio CE)
 eifu.edwards.com + 1 888 570 4016	Follow Instructions for Use on the website	Consulter le mode d'emploi sur le site Web	Siehe Gebrauchs- anweisung auf Website	Siga las instrucciones de uso del sitio web	Seguire le istruzioni per l'uso sul sito Web
	Store in a cool, dry place	Conserver dans un endroit frais et sec	Das Produkt an einem kühlen, trockenen Ort lagern	Guárdese en un lugar fresco y seco	Conservare in un luogo fresco e asciutto
	Model Number	Référence	Modellnummer	Número de modelo	Numero di modello
	Unique device identifier	Identifiant unique du dispositif	Einmalige Produktken- nung	Identificador único del dispositivo	Identificatore univoco del dispositivo

Symbol Legend ■ Légende des symboles ■ Zeichenerklärung ■ Significado de los símbolos
■ Legenda dei simboli ■ Lijst met symbolen ■ Symbolforklaring ■ Symbolförklaring
■ Υπόμνημα συμβόλων ■ Legenda dos símbolos ■ Legenda se symboly ■ Jelmagyarázat
■ Legenda simboli ■ Vysvetlivky k symbolom ■ Symbolforklaring ■ Merkkien selitykset
■ Легенда на символите ■ Legendă de simboluri ■ Sümbolite seletus ■ Simbolių
paaiškinimas ■ Simbolu skaidrojums ■ Sembol Açıklamaları ■ Условные обозначения
■ Legenda sa simbolima ■ Legenda simbola

	English	Français	Deutsch	Español	Italiano
	Importer	Importateur	Importeur	Importador	Importatore
	Do not use if package is damaged and consult instructions for use	Ne pas utiliser si le conditionnement est endommagé et consulter le mode d'emploi	Bei beschädigter Verpackung nicht verwenden und Gebrauchsanweisung beachten	No lo utilice si el envase está dañado y consulte las instrucciones de uso	Non utilizzare se la confezione è danneggiata e consultare le istruzioni per l'uso
	MR Unsafe	Risques en milieu RM	MR-unsicher	No seguro para RM	Non compatibile con RM

	Nederlands	Dansk	Svenska	Ελληνικά	Português
	Uit het zonlicht houden	Beskyttes mod sollys	Skyddas mot solljus	Φυλάσσετε μακριά από την άμεση ηλιακή ακτινοβολία	Mantenha longe da luz solar
	Breekbaar, voorzichtigheid geboden	Skrøbelig, skal håndteres forsigtigt	Ömtåligt, hanteras försiktigt	Εύθραστο: χειριστείτε με προσοχή	Frágil, manuseie com cuidado

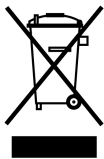
Symbol Legend ■ Légende des symboles ■ Zeichenerklärung ■ Significado de los símbolos
■ Legenda dei simboli ■ Lijst met symbolen ■ Symbolforklaring ■ Symbolförklaring
■ Υπόμνημα συμβόλων ■ Legenda dos símbolos ■ Legenda se symboly ■ Jelmagyarázat
■ Legenda simboli ■ Vysvetlivky k symbolom ■ Symbolforklaring ■ Merkkien selitykset
■ Легенда на символите ■ Legendă de simboluri ■ Sümbolite seletus ■ Simbolių
paaiškinimas ■ Simbolu skaidrojums ■ Sembol Açıklamaları ■ Условные обозначения
■ Legenda sa simbolima ■ Legenda simbola

	Nederlands	Dansk	Svenska	Ελληνικά	Português
	Droog houden	Opbevares tørt	Förvaras torrt	Διατηρείτε στεγνό	Manter seco
	Klein	Lille	Liten	Μικρή	Pequeno
	Gemiddeld	Middel	Medium	Μεσαία	Médio
	Groot	Stor	Stor	Μεγάλη	Grande
	Medisch instrument	Medicinsk udstyr	Medicinteknisk produkt	Ιατροτεχνολογικό προϊόν	Medical Device
	Fabrikant	Producent	Tillverkare	Κατασκευαστής	Fabricante

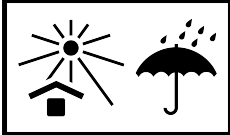
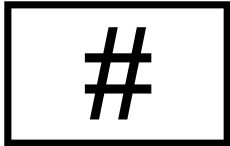
Symbol Legend ■ Légende des symboles ■ Zeichenerklärung ■ Significado de los símbolos
■ Legenda dei simboli ■ Lijst met symbolen ■ Symbolforklaring ■ Symbolförklaring
■ Υπόμνημα συμβόλων ■ Legenda dos símbolos ■ Legenda se symboly ■ Jelmagyarázat
■ Legenda simboli ■ Vysvetlivky k symbolom ■ Symbolforklaring ■ Merkkien selitykset
■ Легенда на символите ■ Legendă de simboluri ■ Sümbolite seletus ■ Simbolių
paaiškinimas ■ Simbolu skaidrojums ■ Sembol Açıklamaları ■ Условные обозначения
■ Legenda sa simbolima ■ Legenda simbola

	Nederlands	Dansk	Svenska	Ελληνικά	Português
	Niet opnieuw gebruiken	Må ikke genbruges	Får inte återanvändas	Μην επαναχρησιμοποιείτε	Não reutilizar
	Fabricagedatum	Fremstillingsdato	Tillverkningsdatum	Ημερομηνία κατασκευής	Data de fabricação
	Lotnummer	Partinummer	Lotnummer	Αριθμός Παρτίδας	Número de lote
	Niet-steriel	Ikke-steril	Icke-steril	Μη αποστειρωμένο	Não estéril
	Geautoriseerde vertegenwoordiger in de Europese Gemeenschap	Autoriseret repræsentant i Det Europæiske Fællesskab	Auktoriserad representant inom Europeiska gemenskapen	Εξουσιοδοτημένος αντιπρόσωπος στην Ευρωπαϊκή Ένωση	Representante autorizado na Comunidade Europeia

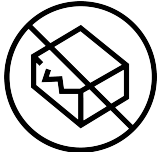
Symbol Legend ■ Légende des symboles ■ Zeichenerklärung ■ Significado de los símbolos
■ Legenda dei simboli ■ Lijst met symbolen ■ Symbolforklaring ■ Symbolförklaring
■ Υπόμνημα συμβόλων ■ Legenda dos símbolos ■ Legenda se symboly ■ Jelmagyarázat
■ Legenda simboli ■ Vysvetlivky k symbolom ■ Symbolforklaring ■ Merkkien selitykset
■ Легенда на символите ■ Legendă de simboluri ■ Sümbolite seletus ■ Simbolių
paaiškinimas ■ Simbolu skaidrojums ■ Sembol Açıklamaları ■ Условные обозначения
■ Legenda sa simbolima ■ Legenda simbola

	Nederlands	Dansk	Svenska	Ελληνικά	Português
	Gescheiden inzameling voor elektrische en elektronische apparatuur in overeenstemming met EG-richtlijn 2012/19/EU.	Separat indsamling af elektrisk og elektronisk udstyr iht. EF-direktivet 2012/19/EU.	Separat insamling för elektrisk och elektronisk utrustning i enlighet med EU-direktiv 2012/19/EU.	Ξεχωριστή συλλογή για ηλεκτρικό και ηλεκτρονικό εξοπλισμό σύμφωνα με την Οδηγία του Ευρωπαϊκού Κοινοβουλίου 2012/19/ΕΕ.	Coleta separada para equipamentos eletrônicos e elétricos em conformidade com a Diretiva CE 2012/19/UE.
	Hoeveelheid	Mængde	Antal	Ποσότητα	Quantidade
	Vervaldatum	Anvendes inden	Sista förbrukningsdag	Ημερομηνία λήξης	Data de vencimento
	Let op	Forsigtig	Var försiktig	Προσοχή	Aviso
	Volg de gebruiksaanwijzing	Følg brugsanvisningen	Följ bruksanvisningen	Ακολουθείτε τις οδηγίες χρήσης	Consulte as Instruções de Uso

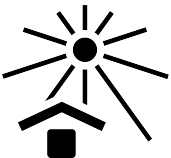
Symbol Legend ■ Légende des symboles ■ Zeichenerklärung ■ Significado de los símbolos
■ Legenda dei simboli ■ Lijst met symbolen ■ Symbolforklaring ■ Symbolförklaring
■ Υπόμνημα συμβόλων ■ Legenda dos símbolos ■ Legenda se symboly ■ Jelmagyarázat
■ Legenda simboli ■ Vysvetlivky k symbolom ■ Symbolforklaring ■ Merkkien selitykset
■ Легенда на символите ■ Legendă de simboluri ■ Sümbolite seletus ■ Simbolių
paaiškinimas ■ Simbolu skaidrojums ■ Sembol Açıklamaları ■ Условные обозначения
■ Legenda sa simbolima ■ Legenda simbola

	Nederlands	Dansk	Svenska	Ελληνικά	Português
Rx only	Let op: Federal (USA) law restricts this device to sale by or on the order of a physician.	Forsigtig: Federal (USA) law restricts this device to sale by or on the order of a physician.	Var försiktig: Federal (USA) law restricts this device to sale by or on the order of a physician.	Προσοχή: Federal (USA) law restricts this device to sale by or on the order of a physician.	Aviso: a lei federal (nos EUA) restringe este aparelho à venda por um médico ou sob sua solicitação.
	Conformité Européenne (CE- markering)	Conformité Européenne (CE-mærke)	Conformité Européenne (CE-märke)	Conformité Européenne (σήμανση CE)	Conformité Européenne (Marca CE)
 eifu.edwards.com + 1 888 570 4016	Volg de gebruiksaan- wijzing op de website	Følg brugsanvisnin- gen på hjemmesiden	Följ bruksanvis- ningen på webbplatsen	Ακολουθείτε τις οδηγίες χρήσης που αναφέρονται στην τοποθεσία web	Siga as instruções de uso no site
	Op een koele en droge plaats bewaren	Skal opbevares køligt og tørt	Förvara svalt och torrt.	Φυλάσσετε σε δροσερό και ξηρό χώρο	Armazenar em um local fresco e seco
	Modelnummer	Modelnummer	Modellnummer	Αριθμός μοντέλου	Número de modelo

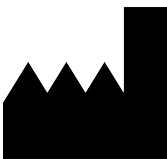
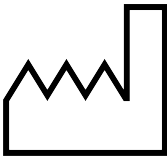
Symbol Legend ■ Légende des symboles ■ Zeichenerklärung ■ Significado de los símbolos
■ Legenda dei simboli ■ Lijst met symbolen ■ Symbolforklaring ■ Symbolförklaring
■ Υπόμνημα συμβόλων ■ Legenda dos símbolos ■ Legenda se symboly ■ Jelmagyarázat
■ Legenda simboli ■ Vysvetlivky k symbolom ■ Symbolforklaring ■ Merkkien selitykset
■ Легенда на символите ■ Legendă de simboluri ■ Sümbolite seletus ■ Simbolių
paaiškinimas ■ Simbolu skaidrojums ■ Sembol Açıklamaları ■ Условные обозначения
■ Legenda sa simbolima ■ Legenda simbola

	Nederlands	Dansk	Svenska	Ελληνικά	Português
	Unieke instrumentidentificatie	Unik udstyrsidentifikation	Unik produktidentifiering	Μοναδικό αναγνωριστικό συσκευής	Identificador único do dispositivo
	Importeur	Importør	Importör	Εισαγωγέας	Importador
	Niet gebruiken als de verpakking is beschadigd en raadpleeg de gebruiksaanwijzing	Må ikke anvendes, hvis emballagen er beskadiget, og se brugsanvisningen	Använd inte om förpackningen är skadad och se bruksanvisningen	Μη χρησιμοποιείτε εάν η συσκευασία έχει υποστεί ζημιά και συμβουλευτείτε τις οδηγίες χρήσης	Não use se a embalagem estiver danificada e consulte as instruções de uso
	MRI-onveilig	MR-usikker	MR-farlig	Μη ασφαλές σε περιβάλλον μαγνητικής τομογραφίας (MR)	Inseguro em ambiente de RM

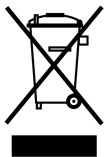
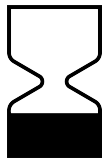
Symbol Legend ■ Légende des symboles ■ Zeichenerklärung ■ Significado de los símbolos
■ Legenda dei simboli ■ Lijst met symbolen ■ Symbolforklaring ■ Symbolförklaring
■ Υπόμνημα συμβόλων ■ Legenda dos símbolos ■ Legenda se symboly ■ Jelmagyarázat
■ Legenda simboli ■ Vysvetlivky k symbolom ■ Symbolforklaring ■ Merkkien selitykset
■ Легенда на символите ■ Legendă de simboluri ■ Sümbolite seletus ■ Simbolių
paaiškinimas ■ Simbolu skaidrojums ■ Sembol Açıklamaları ■ Условные обозначения
■ Legenda sa simbolima ■ Legenda simbola

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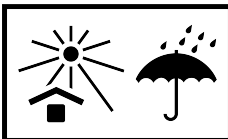
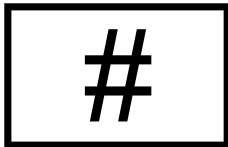
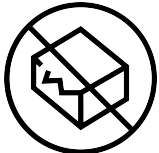
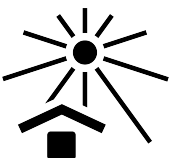
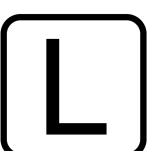
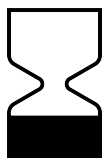
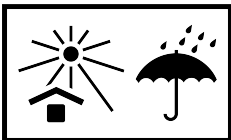
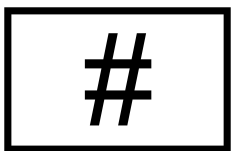
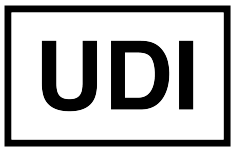
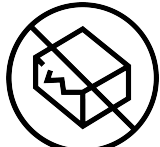
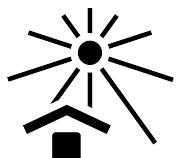
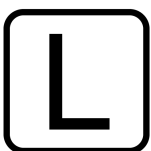
	Suomi	Български	Română	Eesti	Lietuvių
Rx only	Tärkeä huomautus: Federal (USA) law restricts this device to sale by or on the order of a physician.	Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.	Atenție: Federal (USA) law restricts this device to sale by or on the order of a physician.	Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.	Perspėjimas: Federal (USA) law restricts this device to sale by or on the order of a physician.
	Conformité Européenne (CE-merkintä)	Conformité Européenne (CE маркировка)	Conformité Européenne (marcaj CE)	Conformité Européenne (CE-märk)	Conformité Européenne (CE žymė)
 eifu.edwards.com + 1 888 570 4016	Noudata käyttöohjeita , jotka ovat verkkosivustolla	Спазвайте инструкциите за употреба, дадени на уебсайта	Respectați instrucțiunile de utilizare disponibile pe site-ul web	Järgige kasutusjuhendit, mille leiате veebisaidilt	Vadovautis naudojimo instrukcijomis, pateikiamomis svetainėje
	Säilytettävä kuivassa ja viileässä	Да се съхранява на хладно, сухо място	Stocați într-un loc rece și uscat	Säilitage jahedas ja kuivas kohas.	Laikyti vėsioje, sausoje vietoje
	Mallinnumero	Номер на модела	Număr model	Mudeli number	Modelio numeris
	Laitteen yksilöivä tunnus	Уникален идентификатор на изделието	Identificator unic al dispozitivului	Seadme kordumatu identifitseerimistunnus	Unikalasis priemonės identifikatorius

Symbol Legend ■ Légende des symboles ■ Zeichenerklärung ■ Significado de los símbolos
■ Legenda dei simboli ■ Lijst met symbolen ■ Symbolforklaring ■ Symbolförklaring
■ Υπόμνημα συμβόλων ■ Legenda dos símbolos ■ Legenda se symboly ■ Jelmagyarázat
■ Legenda simboli ■ Vysvetlivky k symbolom ■ Symbolforklaring ■ Merkkien selitykset
■ Легенда на символите ■ Legendă de simboluri ■ Sümbolite seletus ■ Simbolių
paaiškinimas ■ Simbolu skaidrojums ■ Sembol Açıklamaları ■ Условные обозначения
■ Legenda sa simbolima ■ Legenda simbola

	Suomi	Български	Română	Eesti	Lietuvių
	Maahantuoja	Вносител	Importator	Importija	Importuotojas
	Älä käyttää, jos pakkaus on vahingoittunut, ja katso käyttöohjeet	Да не се използва, ако опаковката е повредена, и направете справка с инструкциите за употреба	A nu se utiliza dacă ambalajul este deteriorat. Consultați instrucțiunile de utilizare	Ärge kasutage, kui pakend on kahjustatud, ja palun lugege kasutusjuhiseid	Nenaudoti, jei pakuotė pažeista, ir žr. naudojimo instrukcijas
	Ei sovi magneettikuvaukseen	Небезопасно при MR	Incompatibil RM	Ohtlik magnetresonantstomograafias	MR nesaugus

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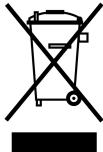
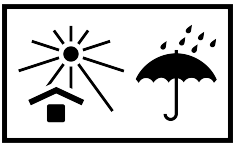
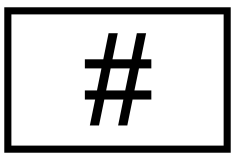
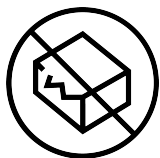
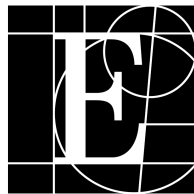
	Latviešu	Türkçe	Русский	Srpski	Hrvatski
Rx only	Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.	Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.	Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.	Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.	Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.
	Conformité Européenne (CE marķējums)	Conformité Européenne (CE İşareti)	Conformité Européenne (маркировка CE)	Conformité Européenne (CE znak)	Conformité Européenne (oznaka CE)
 eifu.edwards.com + 1 888 570 4016	Ievērojiet lietošanas instrukciju, kas pieejama tīmekļa vietnē	Web sitesinde bulunan kullanım talimatlarını takip edin	Следуйте инструкциям по применению на веб-сайте	Pridržavajte se uputstva za upotrebu sa veb-sajta	Pridržavajte se uputa za upotrebu na internetskoj stranici
	Uzglabāt vēsā, sausā vietā	Serin, kuru yerde saklayın	Хранить в прохладном, сухом месте	Čuvati na hladnom i suvom mestu	Pohranite na hladnom, suhom mjestu
	Modeļa numurs	Model Numarası	Номер модели	Broj modela	Broj modela
	Unikāls ierīces identifikators	Benzersiz cihaz tanımlayıcısı	Уникальный идентифика- тор устройства	Jedinstveni identifikator sredstva	Jedinstveni identifikator proizvoda

Symbol Legend ■ Légende des symboles ■ Zeichenerklärung ■ Significado de los símbolos
■ Legenda dei simboli ■ Lijst met symbolen ■ Symbolforklaring ■ Symbolförklaring
■ Υπόμνημα συμβόλων ■ Legenda dos símbolos ■ Legenda se symboly ■ Jelmagyarázat
■ Legenda simboli ■ Vysvetlivky k symbolom ■ Symbolforklaring ■ Merkkien selitykset
■ Легенда на символите ■ Legendă de simboluri ■ Sümbolite seletus ■ Simbolių
paaiškinimas ■ Simbolu skaidrojums ■ Sembol Açıklamaları ■ Условные обозначения
■ Legenda sa simbolima ■ Legenda simbola

	Latviešu	Türkçe	Русский	Srpski	Hrvatski
	Importētājs	İthalatçı	Импортер	Uvoznik	Uvoznik
	Nelietot, ja iepakojums ir bojāts, un skatīt lietošanas instrukciju	Ambalaj hasarlıysa kullanmayın ve kullanım talimatlarına başvurun	Не использовать, если упаковка повреждена; использовать в соответствии с инструкцией	Ne koristite ako je pakovanje oštećeno i pogledajte uputstvo za upotrebu	Nemojte upotrebljavati ako je pakiranje oštećeno i pogledajte upute za upotrebu
	Nedrīkst lietot MR vidē	MR için Güvenli Değildir	Опасно при проведении МРТ	Nije bezbedno za MR	Nije sigurno kod pregleda MR-om

Note: Not all symbols may be included in the labeling of this product. ■ **Remarque :** il est possible que certains symboles n'apparaissent pas sur les étiquettes de ce produit. ■ **Hinweis:** Unter Umständen sind nicht alle Zeichen auf dem Produktetikett aufgeführt. ■ **Nota:** Es posible que no todos los símbolos aparezcan en el etiquetado de este producto. ■ **Nota:** alcuni simboli potrebbero non essere stati inseriti sull'etichetta del prodotto. ■ **Opmerking:** Het label van dit product bevat misschien niet alle symbolen. ■ **Bemærk:** Alle symbolerne er muligvis ikke inkluderet på produktmærkaterne. ■ **Obs!** Alla symboler kanske inte används för märkning av denna produkt. ■ **Σημείωση:** Ενδέχεται να μην συμπεριλαμβάνονται όλα τα σύμβολα στη σήμανση αυτού του προϊόντος. ■ **Observação:** Nem todos os símbolos podem ser incluídos na rotulagem deste produto. ■ **Poznámka:** Štítky tohoto výrobku nemusí obsahovat všechny symboly. ■ **Megjegyzés:** Elképzelhető, hogy a termék címkéjén nem szerepel az összes szimbólum. ■ **Uwaga:** Na etykiety niniejszego produktu mogą nie znajdować się wszystkie symbole. ■ **Poznámka:** Je možné, že na štítku pre tento produkt nie sú uvedené všetky symboly. ■ **Merk:** Alle symboler er kanskje ikke inkludert på produktmerkingen. ■ **Huomautus:** kaikkia symboleja ei välttämättä ole käytetty tämän tuotteen pakkausmerkinnöissä. ■ **Забележка:** Възможно е не всички символи да са включени в етикета на този продукт. ■ **Notă:** este posibil ca nu toate simbolurile să fie incluse pe eticheta acestui produs. ■ **Märkus.** Kõik sümbolid ei pruugi antud toote etiketil esineda. ■ **Pastaba.** Šio gaminio etiketėje gali būti pateikti ne visi simboliai. ■ **Piezīme.** Šī izstrādājuma marķējumā var nebūt iekļauti visi simboli. ■ **Not:** Bu ürünün etiketinde tüm semboller yer almamış olabilir. ■ **Примечание.** На этикетках данного изделия могут указываться не все символы. ■ **Napomena:** Na ambalaži proizvoda se možda ne nalaze svi simboli. ■ **Napomena:** na oznakama ovog proizvoda ne moraju se nalaziti svi simboli.

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Made in Dominican Republic
**CCT Critical Care Technologies,
S.R.L.**
Parque Industrial Itabo
Km 18.5 Carr. Sanchez
Haina, San Cristobal, Dominican
Republic

02/25
10038609002 A / DOC-0250416 A

© Copyright 2025,
Edwards Lifesciences LLC
All rights reserved.

Edwards Lifesciences LLC
One Edwards Way
Irvine, CA 92614 USA

Edwards Lifesciences B.V.
Verlengde Poolseweg 16
4818 CL Breda, Nether-
lands

Telephone 949.250.2500
800.424.3278
FAX 949.250.2525

Web IFU

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Edwards Lifesciences GmbH