



Edwards

## HemoSphere Nano™ Monitor Release Notes

The following is a list of known device implementation anomalies and troubleshooting steps associated with the listed software release version. The software version is listed on the Versions screen (System Settings Screen → System Settings → Versions). These issues are continually updated and compiled as a result of ongoing product improvements.


### HemoSphere Nano™ monitor anomalies for software release version 01.00.007

Known anomaly	Cause	Suggested actions
When repeatedly connecting an incompatible cuff, a critical fault (System Issue. Service Required) occurs. This prevents the monitor from taking measurements.	The cause is an analog to digital conversion failure after the repeated connection of an incompatible cuff.	Disconnect the incompatible cuff, power cycle the monitor, and reconnect a compatible finger cuff.



03/26  
60061460001 A / DOC-0582627 A  
© Copyright 2026, Edwards Lifesciences LLC  
All rights reserved

---

 **Edwards Lifesciences LLC**  
One Edwards Way  
Irvine, CA 92614 USA  
Made in USA

Telephone 949.250.2500  
800.424.3278  
FAX 949.250.2525

Web IFU