Edwards

KONECT RESILIA Aortic Valved Conduit, Model 11060A Accessory Care and Sterilization Instructions for Tray Model TRAY1190

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Rx only

1.0 Tray Model TRAY1190

The Tray Model TRAY1190 is intended to hold Sizers Model 1190 during sterilization and storage and allows for multiple sterilization cycles (re-use) by steam (moist heat). The Instructions For Use supplied with KONECT RESILIA aortic valved conduit (AVC), Model 11060A provide information on the use of this tray.

2.0 How Supplied

The tray is provided non-sterile and must be cleaned and sterilized before each use. It cannot be sterilized in its original packaging.

3.0 Cleaning Instructions

All trays and lids must be cleaned and sterilized prior to each use in accordance with appropriate hospital procedures and the following recommended instructions and parameters.

Accessories should be retired from use when signs of cracking, crazing, or other deterioration is evident. Contact your Edwards Lifesciences sales representative to obtain appropriate replacements.

CAUTION: Examine tray for signs of wear, such as dullness, cracking or crazing. Replace tray if any deterioration is observed.

3.1 Instructions for Manual Cleaning:

1. Rinse the tray under cold, running tap water to remove gross soil.
2. Prepare a neutral pH cleaning solution (such as Enzol or equivalent) per the manufacturer's recommendation.
3. Fully immerse the tray in the cleaning solution and soak for a minimum of 5 minutes.
4. After the 5-minute soak time, use a soft-bristled brush and scrub the tray for a minimum of 2 minutes to remove any visible soil.
5. Remove the tray from the cleaning solution and rinse with cold, running tap water, then in reverse osmosis/deionized water for a minimum of 1 minute.
6. If soil is visible, repeat cleaning process.

3.2 Instructions for Automated Cleaning:

1. Rinse the tray under cold, running tap water to remove gross soil.
2. Prepare a neutral pH cleaning solution (such as Enzol or equivalent) per the manufacturer's recommendation.
3. Fully immerse the tray in the cleaning solution and soak for a minimum of 1 minute. Avoid contact between the sizers.
4. Load the tray with instruments as indicated by the tray markings.
5. Place the loaded tray into the automated washer with the lid detached.
6. Set motor speed to HIGH, if applicable, and select the cycle and cycle parameters per the following recommendations:

PHASE	RECIRCULATION TIME (MINUTES)	TEMPERATURE	DETERGENT TYPE AND CONCENTRATION
Pre-wash	2:00	Cold tap water	N/A
*Enzyme wash	2:00	Hot tap water	Neutral pH enzymatic detergent (Enzol or equivalent)
Wash	2:00	50-60 °C (122-140 °F)	Alkaline detergent (Neodisher Mediclean Forte or equivalent)
Rinse	1:00	60-70 °C (140-158 °F)	N/A
Drying	7:00	90-115 °C (194-239 °F)	N/A

*The enzyme wash phase is optional and may vary based on the washer model.

4.0 Disinfection:

4.1 Instructions for Thermal Disinfection:

1. Load tray with sizers into washer/disinfector with the lid detached.
2. Set the motor speed to HIGH, if applicable, and select the following recommended parameters:

PHASE	RECIRCULATION TIME (MINUTES)	WATER TEMPERATURE	DETERGENT TYPE AND CONCENTRATION
Thermal rinse	5:00	90 °C (194 °F)	N/A

4.2 Instructions for High Level Disinfection:

1. Equilibrate a bath of high level disinfectant (Cidex OPA or equivalent) to a minimum of 20 °C (68 °F), or follow manufacturer's recommendations.
2. Fully immerse the tray in the disinfectant solution, and ensure all air bubbles are removed from the surface of the tray by wiping with a sterile lint-free cloth.
3. Allow the tray to soak for a minimum of 12 minutes.
4. Thoroughly rinse the tray by fully immersing in sterile purified water or equivalent, agitating, and soaking for a minimum of 1 minute.
5. Dry the tray using a sterile lint-free cloth.

5.0 Sterilization Instructions

CAUTION: Do not sterilize any of the accessories in their original packaging. Each institution should use procedures that include biological indicators to determine the effectiveness of the sterilization procedure.

CAUTION: Do not stack trays during sterilization.

The user is responsible for the qualification of any deviations from the recommended method of cleaning or sterilization.

Sterilizer Type	Configuration	Preconditioning	Minimum Temperature	Exposure Time	Minimum Dry Time
Gravity Displacement	Double Wrapped	N/A	132-137 °C (270-279 °F)	15-18 minutes	40 minutes

Sterilizer Type	Configuration	Preconditioning	Minimum Temperature	Exposure Time	Minimum Dry Time
Pre-vacuum	Double Wrapped	4 pulses	132-137 °C (270-279 °F)	3-18 minutes	30 minutes

Examine all instruments prior to use. If particles are visible, repeat the cleaning and sterilization process.

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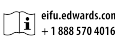
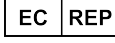
	ISO Reg. No. ¹	English	Français
	2493	Catalogue Number	Référence catalogue
	0434A	Caution	Avertissement
	1641	Consult instructions for use on the website	Consulter le mode d'emploi sur le site Web
	N/A	Quantity	Quantité
	N/A	Size	Taille
Rx only	N/A	Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.	Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.

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■ **Note:** The labeling of this product may not contain every symbol depicted in this legend.

¹ ISO 7000 *Graphical symbols for use on medical equipment - Registered symbols*

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04/2021
10046440001 A / DOC-0172455 A
© Copyright 2021, Edwards Lifesciences LLC
All rights reserved.

Edwards Lifesciences LLC
One Edwards Way
Irvine, CA 92614 USA
Made in USA

Telephone 949.250.2500
800.424.3278
FAX 949.250.2525

Web IFU

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Edwards Lifesciences Services GmbH