Edwards

Bioprosthetic Valve and Annuloplasty Ring Accessories Care and Sterilization Instructions for Sizers

Održavanje dodatne opreme bioprotetičkog zaliska i prstena za anuloplastiku i uputstva za merache veličine

Directory ■ Direktorijum

English (en).....	1
Srpski (sr).....	2
Symbol Legend ■ Legenda sa simbolima.....	4

English

CAUTION: Federal (USA) law restricts this device to sale by or on the order of a physician.

1.0 Accessories

These instructions apply to all sizer models for bioprosthetic valves and annuloplasty rings. The Instructions For Use supplied with Edwards Lifesciences bioprosthetic valves and annuloplasty rings provide information on the use of these accessories.

2.0 How Supplied

The accessories are packaged separately, provided non-sterile and must be cleaned and sterilized before each use. They cannot be sterilized in their original packaging.

3.0 Cleaning Instructions

All sizers must be cleaned, disinfected and sterilized prior to each use in accordance with appropriate hospital procedures and the following recommended instructions and parameters.

Sizers should be retired from use when signs of cracking, crazing, or other deterioration is evident. Contact your Edwards Lifesciences sales representative to obtain appropriate replacements.

CAUTION: Examine sizers for signs of wear, such as dullness, cracking or crazing. Replace accessories if any deterioration is observed.

WARNING: Fragments of the sizers cannot be located by means of an external imaging device.

3.1 Instructions for Manual Cleaning:

1. Rinse the accessories under cold, running tap water to remove gross soil.
2. Prepare a neutral pH cleaning solution (such as Enzol or equivalent) per the manufacturer's

recommendation (1 oz per 1 gallon (30 ml per 3.8 liters) warm tap water).

3. Fully immerse the accessories in the cleaning solution and soak for a minimum of 5 minutes.
4. After the 5-minute soak time, use a soft-bristled brush and scrub the accessories for a minimum of 3 minutes to remove any visible soil.
5. Remove the accessories from the cleaning solution and rinse with cold, running tap water, then in reverse osmosis/deionized water for a minimum of 2 minutes.
6. If soil is visible, repeat the cleaning process.

3.2 Instructions for Automated Cleaning:

1. Rinse the accessories under cold, running tap water to remove gross soil.
2. Prepare a neutral pH cleaning solution (such as Enzol or equivalent) per the manufacturer's recommendation (1 oz per 1 gallon (30 ml per 3.8 liters) warm tap water).
3. Fully immerse the accessories in the cleaning solution and soak for a minimum of 1 minute. Avoid contact between the sizers.
4. Place the loaded tray into the automated washer with the lid detached.
5. Set motor speed to HIGH, if applicable, and select the cycle and cycle parameters per the following recommendations:

Phase	Recirculation Time (Minutes), Set Temperature, Detergent Type
Pre-wash	2:00, Cold tap water, N/A
*Enzyme wash	2:00, Hot tap water, Neutral pH enzymatic detergent (Enzol or equivalent)
Wash	2:00, 50 °C (122 °F), Alkaline detergent (Neodisher MediClean Forte or equivalent)
Rinse	1:00, 60 °C (140 °F), N/A
Drying	7:00, 115 °C (239 °F), N/A

*The enzyme wash phase is optional and may vary based on the washer model.

4.0 Disinfection

4.1 Instructions for Thermal Disinfection:

1. Load tray with sizers into washer/disinfector with the lid detached.

Edwards, Edwards Lifesciences, and the stylized E logo are trademarks of Edwards Lifesciences Corporation. All other trademarks are the property of their respective owners.

2. Set the motor speed to HIGH, if applicable, and select the following recommended parameters:

Phase	Recirculation Time (Minutes), Water Temperature
Thermal rinse	5:00, 90 °C (194 °F)

4.2 Instructions for High-Level Disinfection:

1. Equilibrate a bath of high-level disinfectant (Cidex OPA or equivalent) to a minimum of 20 °C (68 °F), or follow manufacturer's recommendations.
2. Fully immerse accessories in the disinfectant solution, and ensure all air bubbles are removed from the surface by wiping with a sterile, lint-free cloth.
3. Allow the accessories to soak for a minimum of 15 minutes.
4. Thoroughly rinse the accessories by fully immersing in sterile, purified water or equivalent, agitating and soaking for a minimum of 1 minute. Repeat 3 times using fresh, purified water after each rinse.
5. Dry the accessories using a sterile, lint-free cloth.

5.0 Sterilization Instructions

Sizers must be disassembled from any threaded handles before sterilization.

CAUTION: Do not sterilize any of the accessories in their shipping containers. Sizers must be removed from their plastic pouches prior to sterilization. Each institution should use procedures that include biological indicators to determine the effectiveness of the sterilization procedure.

All accessory models can be sterilized using the following recommended autoclave sterilization methods:

Sterilizer Type	Gravity steam sterilization
Configuration	Double Wrapped
Preconditioning	N/A
Minimum Temperature	132 °C (270 °F)
Maximum Temperature	137 °C (279 °F)
Exposure Time	15 minutes
Minimum Dry Time	50 minutes
Minimum Cool Time	60 minutes

Sterilizer Type	Pre-vacuum
Configuration	Double Wrapped
Preconditioning	4 pulses
Minimum Temperature	132 °C (270 °F)
Maximum Temperature	137 °C (279 °F)
Exposure Time	4 minutes
Minimum Dry Time	45 minutes
Minimum Cool Time	15 minutes

The user is responsible for the qualification of any deviations from the recommended method of cleaning or sterilization.

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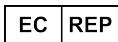
	ISO Reg. No. ¹	English	Srpski
	N/A	Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.	Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.
	2492	Batch Code	Šifra serije
	N/A	Authorized representative in the European Community	Ovlašćeni predstavnik u Evropskoj zajednici
	2609	Non-sterile	Nesterilno
	3082	Manufacturer	Proizvođač
	2497	Date of manufacture	Datum proizvodnje

¹ ISO 7000 Graphical symbols for use on medical equipment - Registered symbols ■ ¹ ISO 7000 Grafički simboli za upotrebu na medicinskoj opremi – Registrovani simboli

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2025-07
60052895001 A
© Copyright 2025, Edwards Lifesciences LLC
All rights reserved.

Edwards Lifesciences LLC
One Edwards Way
Irvine, CA 92614 USA

Telephone 949.250.2500
800.424.3278
FAX 949.250.2525

Web IFU

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Edwards Lifesciences GmbH