Edwards

KONECT RESILIA Aortic Valved Conduit, Model 11060A Accessory Care and Sterilization Instructions for Sizer Model 1190

Directory	
English (en).....	1
Français (fr).....	4
Deutsch (de).....	6
Español (es).....	8
Italiano (it).....	10
Nederlands (nl).....	12
Dansk (da).....	14
Svenska (sv).....	16
Ελληνικά (el).....	18
Português (pt).....	20
Česky (cs).....	22
Magyar (hu).....	24
Polski (pl).....	26
Slovensky (sk).....	28
Norsk (no).....	30
Suomi (fi).....	32
Български (bg).....	34
Română (ro).....	36
Eesti (et).....	38
Lietuvių (lt).....	40
Latviešu (lv).....	42
Türkçe (tr).....	44
Hrvatski (hr).....	46
Symbol Legend ■ Légende des symboles ■ Zeichenerklärung ■ Significado de los símbolos ■ Legenda dei simboli ■ Lijst met symbolen ■ Symbolforklaring ■ Symbolförklaring ■ Υπόμνημα συμβόλων ■ Legenda de símbolos ■ Legenda se symboly ■ Jelmagyarázat ■ Legenda simboli ■ Vysvetlivky k symbolom ■ Symbolforklaring ■ Merkkien selitykset ■ Легенда на символите ■ Legendă de simboluri ■ Sümbolite seletus ■ Simbolių paaiškinimas ■ Simbolu skaidrojums ■ Sembol Açıklamaları ■ Legenda simbola.....	48

English

KONECT RESILIA Aortic Valved Conduit, Model 11060A Accessory Care and Sterilization Instructions for Sizer Model 1190

Rx only

1.0 Sizer Model 1190

The Sizer Model 1190 is intended to facilitate the selection of the appropriate size of KONECT RESILIA aortic valved conduit (AVC), Model 11060A. Sizers are available in sizes 19, 21, 23, 25, 27 and 29 mm. Sizer Model 1190 allows for direct observation of KONECT AVC fit within the annulus after the native leaflets have been removed or prior prosthetic valve is removed.

The Instructions For Use supplied with KONECT RESILIA AVC provide information on the use of these sizers.

2.0 How Supplied

The sizers are packaged separately, provided non-sterile and must be cleaned and sterilized before each use. They cannot be sterilized in their original packaging.

3.0 Cleaning Instructions

All sizers must be cleaned and sterilized prior to each use in accordance with appropriate hospital procedures and the following recommended instructions and parameters. Accessories should be retired from use when signs of cracking, crazing, or other deterioration is evident. Contact your Edwards Lifesciences sales representative to obtain appropriate replacements.

CAUTION: Examine sizers for signs of wear, such as dullness, cracking or crazing. Replace sizer if any deterioration is observed.

WARNING: Fragments of the sizers cannot be located by means of an external imaging device.

3.1 Instructions for Manual Cleaning

1. Rinse the sizers under cold, running tap water to remove gross soil.
2. Prepare a neutral pH cleaning solution (such as Enzol or equivalent) per the manufacturer's recommendation.
3. Fully immerse the sizers in the cleaning solution and soak for a minimum of 5 minutes. Avoid contact between the sizers.
4. After the 5 minute soak time, use a soft-bristled brush and scrub the sizers for a minimum of 2 minutes to remove any visible soil.
5. Remove the sizers from the cleaning solution and rinse with cold, running tap water, then in reverse osmosis/deionized water for a minimum of 1 minute.
6. If soil is visible, repeat cleaning process.

3.2 Instructions for Automated Cleaning

1. Rinse the sizers under cold, running tap water to remove gross soil.
2. Prepare a neutral pH cleaning solution (such as Enzol or equivalent) per the manufacturer's recommendation.
3. Fully immerse the sizers in the cleaning solution and soak for a minimum of 1 minute. Avoid contact between the sizers.
4. Load the sizers into the TRAY1190 tray as indicated on the base of the tray.
5. Place the loaded tray into the automated washer with the lid detached.
6. Set motor speed to HIGH, if applicable, and select the cycle and cycle parameters per the following recommendations:

PHASE	RECIRCULATION TIME (MINUTES)	TEMPERATURE	DETERGENT TYPE AND CONCENTRATION
Pre-wash	2:00	Cold tap water	N/A
*Enzyme wash	2:00	Hot tap water	Neutral pH enzymatic detergent (Enzol or equivalent)
Wash	2:00	50-60 °C (122-140 °F)	Alkaline detergent (Neodisher Mediclean Forte or equivalent)
Rinse	1:00	60-70 °C (140-158 °F)	N/A
Drying	7:00	90-115 °C (194-239 °F)	N/A

*The enzyme wash phase is optional and may vary based on the washer model.

4.0 Disinfection

4.1 Instructions for Thermal Disinfection

1. Load tray with sizers into washer/disinfector with the lid detached.
2. Set the motor speed to HIGH, if applicable, and select the following recommended parameters:

PHASE	RECIRCULATION TIME (MINUTES)	WATER TEMPERATURE	DETERGENT TYPE AND CONCENTRATION
Thermal rinse	5:00	90 °C (194 °F)	N/A

4.2 Instructions for High Level Disinfection

1. Equilibrate a bath of high level disinfectant (Cidex OPA or equivalent) to a minimum of 20 °C (68 °F), or follow manufacturer's recommendations.
2. Fully immerse the sizers in the disinfectant solution, and ensure all air bubbles are removed from the surface of the sizers by wiping with a sterile lint-free cloth.
3. Allow the sizers to soak for a minimum of 12 minutes.
4. Thoroughly rinse the sizers by fully immersing in sterile purified water or equivalent, agitating, and soaking for a minimum of 1 minute.
5. Dry the sizers using a sterile lint-free cloth.

5.0 Sterilization Instructions

CAUTION: Do not sterilize any of the accessories in their original packaging. Each institution should use procedures that include biological indicators to determine the effectiveness of the sterilization procedure.

CAUTION: Do not stack trays during sterilization.

The user is responsible for the qualification of any deviations from the recommended method of cleaning or sterilization.

Sterilizer Type	Configuration	Preconditioning	Minimum Temperature	Exposure Time	Minimum Dry Time
Gravity Displacement	Double Wrapped	N/A	132-137 °C (270-279 °F)	15-18 minutes	40 minutes
Pre-vacuum	Double Wrapped	4 pulses	132-137 °C (270-279 °F)	3-18 minutes	30 minutes

Examine all instruments prior to use. If particles are visible, repeat the cleaning and sterilization process.

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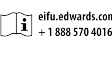
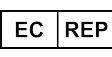
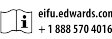
	ISO Reg. No. ¹	English	Français	Deutsch	Español	Italiano
	2493	Catalogue Number	Référence catalogue	Artikelnummer	Número de catálogo	Numero di catalogo
	0434A	Caution	Avertissement	Vorsicht	Aviso	Attenzione
	1641	Consult instructions for use on the website	Consulter le mode d'emploi sur le site Web	Gebrauchsanweisung auf der Website beachten	Consulte las instrucciones de uso en el sitio web	Consultare le istruzioni per l'uso sul sito Web
	N/A	Quantity	Quantité	Menge	Cantidad	Quantità
	N/A	Size	Taille	Größe	Tamaño	Misura
Rx only	N/A	Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.	Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.	Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.	Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.	Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.
	2609	Non-sterile	Non stérile	Unsteril	No estéril	Non sterile
	2606	Do not use if package is damaged	Ne pas utiliser si le conditionnement est endommagé	Bei beschädigter Verpackung nicht verwenden	No lo utilice si el envase está dañado	Non utilizzare se la confezione è danneggiata
	N/A	Authorized representative in the European Community	Représentant autorisé dans la Communauté européenne	Bevollmächtigter in der Europäischen Gemeinschaft	Representante autorizado en la Comunidad Europea	Rappresentante autorizzato nella Comunità Europea
	2492	Lot Number	N° du lot	Chargenbezeichnung	Número de lote	Numero di lotto
	3082	Manufacturer	Fabricant	Hersteller	Fabricante	Produttore
	2497	Date of manufacture	Date de fabrication	Herstellungsdatum	Fecha de fabricación	Data di produzione

	ISO Reg. No. ¹	Nederlands	Dansk	Svenska	Ελληνικά	Português
	2493	Catalogusnummer	Katalognummer	Katalognummer	Αριθμός καταλόγου	Número de catálogo
	0434A	Let op	Forsigtig	Var försiktig	Προσοχή	Aviso
	1641	Raadpleeg de gebruiksaanwijzing op de website	Se brugsanvisningen på webstedet	Se bruksanvisningen på webbplatsen	Συμβουλευτείτε τις οδηγίες χρήσης στον ιστότοπο	Consultar as instruções de utilização no site
	N/A	Hoeveelheid	Mængde	Antal	Ποσότητα	Quantidade

Symbol Legend ■ Légende des symboles ■ Zeichenerklärung ■ Significado de los símbolos ■ Legenda dei simboli ■ Lijst met symbolen ■ Symbolforklaring ■ Symbolförklaring ■ Υπόμνημα συμβόλων ■ Legenda de símbolos ■ Legenda se symboly ■ Jelmagyarázat ■ Legenda simboli ■ Vysvetlivky k symbolom ■ Symbolforklaring ■ Merkkien selitykset ■ Легенда на символите ■ Legendă de simboluri ■ Sümbolite seletus ■ Simbolių paaiškinimas ■ Simbolu skaidrojums ■ Sembol Açıklamaları ■ Legenda simbola

	ISO Reg. No. ¹	Nederlands	Dansk	Svenska	Ελληνικά	Português
	N/A	Afmetingen	Størrelse	Storlek	Μέγεθος	Tamanho
Rx only	N/A	Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.	Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.	Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.	Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.	Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.
	2609	Niet-steriel	Ikke-steril	Icke-steril	Μη αποστειρωμένο	Não esterilizado
	2606	Niet gebruiken als de verpakking is beschadigd	Må ikke anvendes, hvis emballagen er beskadiget	Använd inte om förpackningen är skadad	Μη χρησιμοποιείτε εάν η συσκευασία έχει υποστεί ζημιά	Não utilizar se a embalagem estiver danificada
	N/A	Geautoriseerde vertegenwoordiger in de Europese Gemeenschap	Autoriseret repræsentant i Det Europæiske Fællesskab	Auktoriserad representant inom Europeiska gemenskapen	Εξουσιοδοτημένος αντιπρόσωπος στην Ευρωπαϊκή Ένωση	Representante autorizado na Comunidade Europeia
	2492	Lotnummer	Partinummer	Lotnummer	Αριθμός παρτίδας	Número de lote
	3082	Fabrikant	Producent	Tillverkare	Κατασκευαστής	Fabricante
	2497	Fabricagedatum	Fremstillingsdato	Tillverkningsdatum	Ημερομηνία κατασκευής	Data de fabrico

	ISO Reg. No. ¹	Česky	Magyar	Polski	Slovensky	Norsk
	2493	Katalogové číslo	Katalógusszám	Numer katalogowy	Katalógové číslo	Katalognummer
	0434A	Výstraha	Vigyázat!	Przestroga	Upozornenie	Forsiktig
	1641	Postupujte podle návodu k použití, který naleznete na webových stránkách	Olvasa el a használati utasítást a weboldalon	Należy zapoznać się z instrukcją użycia na stronie internetowej	Prečítajte si návod na použitie na webovej stránke	Se bruksanvisningen på nettstedet
	N/A	Množství	Mennyiség	Ilość	Množstvo	Antall
	N/A	Velikost	Méret	Rozmiar	Veľkosť	Størrelse
Rx only	N/A	Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.	Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.	Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.	Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.	Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.
	2609	Nesterilní	Nem steril	Niejałowy	Nesterilné	Usteril

Symbol Legend ■ Légende des symboles ■ Zeichenerklärung ■ Significado de los símbolos ■ Legenda dei simboli ■ Lijst met symbolen ■ Symbolforklaring ■ Symbolförklaring ■ Υπόμνημα συμβόλων ■ Legenda de símbolos ■ Legenda se symboly ■ Jelmagyarázat ■ Legenda simboli ■ Vysvetlivky k symbolom ■ Symbolforklaring ■ Merkkien selitykset ■ Легенда на символите ■ Legendă de simboluri ■ Sümbolite seletus ■ Simbolių paaiškinimas ■ Simbolu skaidrojums ■ Sembol Açıklamaları ■ Legenda simbola

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	ISO Reg. No. ¹	Suomi	Български	Română	Eesti	Lietuvių
	2493	Luettelonumero	Каталожен номер	Număr de catalog	Kataloogi number	Katalogo numeris
	0434A	Tärkeä huomautus	Внимание	Atenție	Ettevaatust	Perspėjimas
	1641	Katso käyttöohjeet verkkosivustolta	Направете справка с инструкциите за употреба в уебсайта	Consultați instrucțiunile de utilizare de pe site-ul web	Palun lugege kasutusjuhiseid veebisaidilt	Žr. naudojimo instrukcijas interneto svetainėje
	N/A	Määrä	Количество	Cantitate	Kogus	Kiekis
	N/A	Koko	Размер	Mărimē	Suurus	Dydis
Rx only	N/A	Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.	Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.	Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.	Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.	Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.
	2609	Epästeriili	Нестерилно	Nesteril	Mittesteriilne	Nesterilus
	2606	Älä käytä, jos pakkaus on vahingoittunut	Да не се използва, ако опаковката е повредена	A nu se utiliza dacă ambalajul este deteriorat	Ärge kasutage, kui pakend on kahjustatud	Nenaudoti, jei pakuotė pažeista
	N/A	Valtuutettu edustaja Euroopan yhteisössä	Упълномощен представител в Европейската общност	Reprezentant autorizat în Comunitatea Europeană	Volitatud esindaja Euroopa Ühenduses	Igaliotasis atstovas Europos Bendrijoje
	2492	Eränumero	Партиден номер	Număr de lot	Partii number	Partijos numeris

Symbol Legend ■ Légende des symboles ■ Zeichenerklärung ■ Significado de los símbolos ■ Legenda dei simboli ■ Lijst met symbolen ■ Symbolforklaring ■ Symbolförklaring ■ Υπόμνημα συμβόλων ■ Legenda de símbolos ■ Legenda se symboly ■ Jelmagyarázat ■ Legenda simboli ■ Vysvětlivky k symbolom ■ Symbolforklaring ■ Merkkien selitykset ■ Легенда на символите ■ Legendă de simboluri ■ Sümbolite seletus ■ Simbolių paaiškinimas ■ Simbolu skaidrojums ■ Sembol Açıklamaları ■ Legenda simbola

	ISO Reg. No. ¹	Suomi	Български	Română	Eesti	Lietuvių
	3082	Valmistaja	Производител	Producător	Tootja	Gamintojas
	2497	Valmistuspäivämäärä	Дата на производство	Data fabricației	Tootmiskuupäev	Pagaminimo data

	ISO Reg. No. ¹	Latviešu	Türkçe	Hrvatski
REF	2493	Kataloga numurs	Katalog Numarası	Kataloški broj
	0434A	Uzmanību!	Dikkat	Oprez
	1641	Skatīt lietošanas instrukciju tīmekļa vietnē	Web sitesindeki kullanım talimatlarına başvurun	Pogledajte upute za upotrebu na internetskoj stranici
#	N/A	Daudzums	Miktar	Količina
SZ	N/A	Izmērs	Boyut	Veličina
Rx only	N/A	Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.	Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.	Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.
	2609	Nesterils	Steril değildir	Nesterilno
	2606	Nelietot, ja iepakojums ir bojāts	Ambalaj hasarlıysa kullanmayın	Nemojte upotrebljavati ako je pakiranje oštećeno
EC REP	N/A	Pilnvarotais pārstāvis Eiropas Kopienā	Avrupa Topluluğu'nda Yetkili Temsilci	Ovlašteni predstavnik u Europskoj zajednici
LOT	2492	Partijas numurs	Lot Numarası	Broj serije
	3082	Ražotājs	Üretici	Proizvođač
	2497	Izgatavošanas datums	Üretim tarihi	Datum proizvodnje

Note: The labeling of this product may not contain every symbol depicted in this legend.

¹ ISO 7000 Graphical symbols for use on medical equipment - Registered symbols

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¹ ISO 7000 *Graphical symbols for use on medical equipment - Registered symbols*

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© Copyright 2024, Edwards Lifesciences LLC
All rights reserved.
Edwards Lifesciences GmbH
Edwards Lifesciences LLC