Edwards

Directory	
English (en).....	1
Français (fr).....	3
Deutsch (de).....	4
Español (es).....	6
Italiano (it).....	7
Nederlands (nl).....	9
Dansk (da).....	10
Svenska (sv).....	12
Ελληνικά (el).....	13
Português (pt).....	15
Česky (cs).....	16
Magyar (hu).....	18
Polski (pl).....	19
Slovensky (sk).....	21
Norsk (no).....	22
Suomi (fi).....	24
Български (bg).....	25
Română (ro).....	27
Eesti (et).....	28
Lietuvių (lt).....	30
Latviešu (lv).....	31
Türkçe (tr).....	33
Русский (ru).....	34
Hrvatski (hr).....	36
Figures ■ Figures ■ Abbildungen ■ Figuras ■ Figure ■ Afbeeldingen ■ Figurer ■ Figurer ■ Εικόνας ■ Figuras ■ Obrázky ■ Ábrák ■ Rysunki ■ Obrázky ■ Figurer ■ Kuvat ■ Фигури ■ Figuri ■ Joonised ■ Paveikslėliai ■ Attēli ■ Şekiller ■ Рисунки ■ Slike.....	38
Symbol Legend ■ Légende des symboles ■ Zeichenerklärung ■ Significado de los símbolos ■ Legenda dei simboli ■ Lijst met symbolen ■ Symbolforklaring ■ Symbolforklaring ■ Υπόμνημα συμβόλων ■ Legenda de símbolos ■ Legenda se symboly ■ Jelmagyarázás ■ Legenda simboli ■ Vysvetlivky k symbolom ■ Symbolforklaring ■ Merkkien selitykset ■ Легенда на символите ■ Legendă de simboluri ■ Sümbolite seletus ■ Simbolių paaiškinimas ■ Simbolu skaidrojums ■ Sembol Açıklamaları ■ Условные обозначения ■ Legenda simbola.....	40

English

Rx only

Edwards Reusable Accessories

Reusable Platform (Model 10000UP), Reusable Plate (Model 10000PT), and Reusable Cradle (Model 10000CR)

Instructions for Use

1.0 Introduction

The Edwards reusable platform, reusable plate, and reusable cradle are reusable, non-sterile accessories indicated for use with compatible Edwards transcatheter cardiac therapies. The reusable platform and reusable plate are non-patient contacting and are intended to aid the positioning and stabilization of delivery systems during intra-cardiac procedures. The reusable platform is height and angle adjustable to accommodate the procedural needs for each system (see Figure 1). The reusable cradle is non-patient contacting and is intended to aid the positioning and stabilization of delivery systems during intra-cardiac procedures (see Figure 1).

The Edwards reusable platform, reusable plate, and reusable cradle are intended to be used in a cardiac procedure room (e.g. catheterization laboratory, hybrid room, etc.).

The Edwards reusable platform, reusable plate, and reusable cradle are intended to be used by physicians and support staff trained to these instructions for use.

2.0 Intended Use

The reusable platform is intended to aid the positioning and stabilization of delivery systems during intra-cardiac procedures.

The reusable plate is intended to aid the positioning and stabilization of delivery systems during intra-cardiac procedures.

The reusable cradle is intended to aid the positioning and stabilization of delivery systems during intra-cardiac procedures.

3.0 Cautions

Read all indications and precautions in these Instructions for Use before using the reusable platform, reusable plate, and reusable cradle. The Instructions for Use is available electronically, via www.eifu.edwards.com. You may also request a copy by calling Edwards Headquarters at +1.888.570.4016 or by contacting your local clinical representative.

The reusable platform, reusable plate, and reusable cradle are NON-STERILE; introduction of the reusable platform, reusable plate, and reusable cradle into the sterile field may result in infection. Prior to use, cleaning must be performed according to the instructions provided in this document (see Sections 7.3 and 7.4).

Do not use metallic brushes, scrub pads, or other abrasive cleaning aids when cleaning the devices. They can cause permanent device damage.

4.0 Limitations on Reprocessing

Proper reprocessing has a minimal effect on the devices.

End of life is normally determined by wear and damage due to use (see Section 7.5 for inspection instructions). Do not use the devices if they are damaged.

These reprocessing instructions have been validated as being capable of preparing the reusable platform, reusable plate, and reusable cradle for non-sterile procedural use. It is the responsibility of the user/hospital/health care provider to ensure that reprocessing is performed using the appropriate equipment and materials, and that personnel have been adequately trained to the reprocessing instructions in order to achieve the desired result; this normally requires that equipment and processes are validated and routinely monitored. Any deviation by the user/hospital/health care provider from these instructions should be evaluated for effectiveness to avoid potential adverse consequences.

5.0 How Supplied

The reusable platform, reusable plate, and reusable cradle are supplied non-sterile.

The reusable platform, reusable plate, and reusable cradle require no assembly or disassembly for procedural use, cleaning, or storage.

6.0 Directions for Use

6.1 Preparation

Prior to use, cleaning must be performed according to the instructions provided in this document (see Sections 7.3 and 7.4).

Step	Procedure
1	Place reusable platform on a flat surface with the top of the platform facing down. 
2	Unfold the reusable platform's legs until there are audible clicks. 

Step	Procedure
3	Flip reusable platform with the top of the platform facing up. Ensure that leg extension knobs are facing user.
	
4	Adjust platform height using the leg extension knobs to ensure clearance between the patient's wrapped legs and the bottom of the reusable platform.

6.2 Use

Step	Procedure
1	Wrap the patient's legs with towels or similar material.
2	Place the reusable plate under the patient's legs.
3	Place the reusable platform over the patient's leg and on top of the reusable plate according to the compatible system IFU and ensure that the leg extension knobs are facing user.
4	Adjust platform height and/or tilt angle as needed according to the compatible system IFU by rotating the leg extension knobs.
5	Place the reusable cradle on the reusable platform according to the compatible system IFU. Push down on the clip until there is an audible click (see Figure 2).

7.0 Reprocessing Instructions

7.1 Equipment and Materials

Wear appropriate protective equipment (gloves, eye protection, etc.) when reprocessing the devices.

Use only 70% isopropyl alcohol for cleaning.

Usage of 70% isopropyl alcohol for cleaning should be consistent with the manufacturer's instructions for use.

7.2 Preparation before processing

Step	Procedure
1	Place the reusable platform, reusable plate, and reusable cradle separately on a flat surface.
2	Ensure the reusable platform legs are unfolded prior to cleaning. Rotate the leg extension knobs clockwise to fully extend both sets of leg extensions (see Figure 3).

7.3 Cleaning

Step	Procedure
1	Reprocess devices as soon as reasonably practical after each use and prior to each reuse.
2	Clean the reusable platform, reusable plate, and reusable cradle by wiping with 70% isopropyl alcohol to remove any visible soil. Ensure all exposed surfaces are wiped. Spraying 70% isopropyl alcohol on exposed surfaces is acceptable.
3	Visually inspect devices for any remaining soil in a well-lit environment. If any device is determined not to be clean, repeat the previous cleaning steps or safely dispose of the device.

7.4 Drying

Step	Procedure
1	If devices are still wet, dry the reusable platform, reusable plate, and reusable cradle with a soft, lint-free cloth after confirming that all visible soil is removed.

7.5 Maintenance, inspection, and testing

Step	Procedure
1	Inspect the reusable platform, reusable plate, and reusable cradle for any unremovable biohazardous stains.
2	Inspect the reusable platform to verify the leg extension knobs fully extend and retract the leg extensions.
3	Inspect the reusable platform to verify the legs remain locked when unfolded.
4	Inspect the reusable platform to verify the leg unlock buttons allow the legs to be folded.
5	Inspect the reusable cradle to verify that there is no damage to the clip.
6	Inspect the reusable platform, reusable plate, and reusable cradle for cracking, sharp edges and/or gross damage in a well-lit environment. Ensure any visible damage does not impact functionality. Normal wear and tear is expected due to repeated use.
7	Do not use device(s) if any inspections are unacceptable.

8.0 Storage

Step	Procedure
1	After cleaning and drying (see Sections 7.3 and 7.4), rotate the reusable platform leg extension knobs counterclockwise to fully retract both sets of leg extensions.
2	Press the leg unlock buttons to unlock and fold each set of legs into the bottom of the reusable platform into the storage configuration (see Figure 1).
3	Store the devices in a cool, dry place.

9.0 Device Disposal

Damaged devices may be handled and disposed of in the same manner as hospital waste in accordance with local regulations as there are no special risks related to the disposal of these devices.

10.0 Basic Unique Device Identification-Device Identifier (UDI-DI)

The Basic UDI-DI is the access key for device-related information entered in the Eudamed. The following table contains the Basic UDI-DIs for the reusable accessories:

Product	Model	Basic UDI-DI
Reusable platform	10000UP	0690103D004REU000YA
Reusable plate	10000PT	0690103D004REU000YA
Reusable cradle	10000CR	0690103D004REU000YA

11.0 Reporting

Users should report any serious incidents to the manufacturer and the Competent Authority of the Member State in which the user is established.

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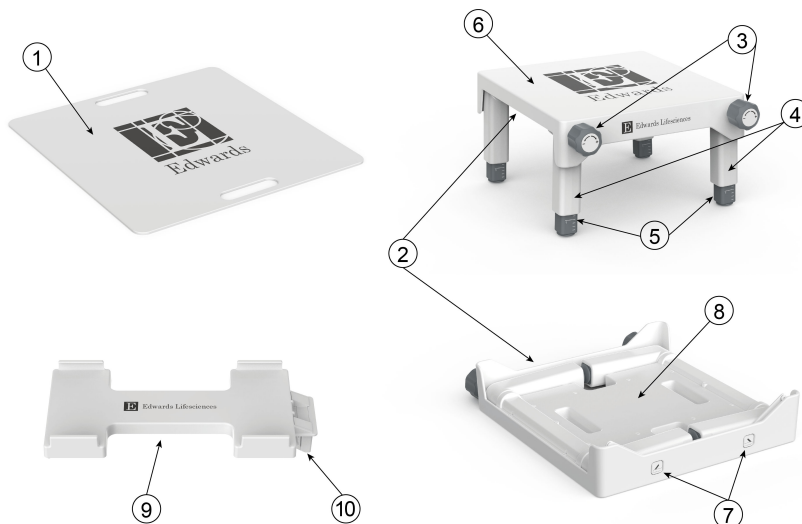
Figure 1: Model 10000UP reusable platform, Model 10000PT reusable plate, and Model 10000CR reusable cradle

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Figure 2: Model 10000CR reusable cradle assembled on Model 10000UP reusable platform and Model 10000PT reusable plate

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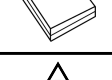
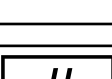
Figure 3: Fully extended configuration of Model 10000UP reusable platform

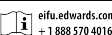
- Figure 3 : Configuration d'extension complète de la plateforme réutilisable en modèle 10000UP
- Abbildung 3: Wiederverwendbare Plattform, Modell 10000UP, vollständig ausgeklappt und ausgefahren
- Figura 3: Configuración totalmente extendida de la plataforma reutilizable modelo 10000UP
- Figura 3: Configurazione completamente allungata del modello 10000UP supporto riutilizzabile
- Afbeelding 3: Volledig verlengde configuratie van model 10000UP herbruikbaar platform
- Figur 3: Helt udfoldet konfiguration af model 10000UP genanvendelig platform
- Figur 3: Helt förlängd konfiguration av modell 10000UP återanvändbar plattform
- Εικόνα 3: Διαμόρφωση πλήρους προέκτασης της επαναχρησιμοποίησης πλατφόρμας, Μοντέλο 10000UP
- Figura 3: Configuração totalmente estendida da plataforma reutilizável modelo 10000UP
- Obrázek 3: Plně rozložená konfigurace opakované použitelné platformy, model 10000UP
- 3. ábra: Az 10000UP típusú újraterhelhető platform teljesen kitolt konfigurációban
- Rysunek 3: W pełni rozłożona konfiguracja modelu 10000UP platformy wielokrotnego użytku
- Obrázok 3: Úplne vysunutá konfigurácia opakovane použiteľnej platformy (model 10000UP)
- Figur 3: Fullt forlenget konfigurasjon av modell 10000UP gjenbrukbar plattform
- Kuva 3: Mallin 10000UP uudelleenkäytettävän järjestelmän täysin pidennetty kokoonpano
- Фигура 3: Напълно разгъната конфигурация на платформа за многократна употреба модел 10000UP
- Figura 3: Configurație complet extinsă a platformei reutilizabile model 10000UP
- Joonis 3: Mudeli 10000UP korduvkasutatava platvormi vaade täielikult avatuna
- 3 pav. 10000UP modelio daugkartinio naudojimo platforma su iki galo pailgintomis kojelėmis
- 3. attēls. Pilnībā paplašināta modeļa 10000UP atkārtoti izmantojamas platformas konfigurācija
- Şekil 3: Model 10000UP yeniden kullanılabilir platformun tamamen uzatılmış biçimi
- Рис. 3. Многоразовая платформа модели 10000UP в полностью развернутой конфигурации
- Slika 3: Potpuno izvučena konfiguracija modela 10000UP platforme za višekratnu uporabu

Symbol Legend ■ Légende des symboles ■ Zeichenerklärung ■ Significado de los símbolos ■ Legenda dei simboli ■ Lijst met symbolen ■ Symbolforklaring ■ Symbolförklaring ■ Υπόμνημα συμβόλων ■ Legenda de símbolos ■ Legenda se symboly ■ Jelmagyarázat ■ Legenda symboli ■ Vysvětlivky k symbolom ■ Symbolforklaring ■ Merkkien selitykset ■ Легенда на символите ■ Legendă de simboluri ■ Sümbolite seletus ■ Simbolių paaiškinimas ■ Simbolu skaidrojums ■ Sembol Açıklamaları ■ Условные обозначения ■ Legenda simbola

	English	Français	Deutsch	Español	Italiano
	Model Number	Référence	Modellnummer	Número de modelo	Numero di modello
	Catalogue Number	Référence catalogue	Artikelnummer	Número de catálogo	Numero di catalogo
	Lot Number	N° du lot	Chargenbezeichnung	Número de lote	Numero di lotto
	Quantity	Quantité	Menge	Cantidad	Quantità
	Contents	Contenu	Inhalt	Contenido	Contenuti
	Caution	Avertissement	Vorsicht	Aviso	Attenzione
	Consult instructions for use or consult electronic instructions for use	Consultez le mode d'emploi sur papier ou le mode d'emploi au format électronique	Gebrauchsanweisung beachten oder elektronische Gebrauchsanweisung beachten	Consulte las instrucciones de uso o las instrucciones de uso electrónicas	Consultare le istruzioni per l'uso o le istruzioni per l'uso elettroniche
	Do not use if package is damaged and consult instructions for use	Ne pas utiliser si le conditionnement est endommagé et consulter le mode d'emploi	Bei beschädigter Verpackung nicht verwenden und Gebrauchsanweisung beachten	No lo utilice si el envase está dañado y consulte las instrucciones de uso	Non utilizzare se la confezione è danneggiata e consultare le istruzioni per l'uso
	Store in a cool, dry place	Conserver dans un endroit frais et sec	Das Produkt an einem kühlen, trockenen Ort lagern	Guárdese en un lugar fresco y seco	Conservare in un luogo fresco e asciutto
	Medical device	Dispositif médical	Medizinprodukt	Producto sanitario	Dispositivo medico
	Consult instructions for use on the website	Consulter le mode d'emploi sur le site Web	Gebrauchsanweisung auf der Website beachten	Consulte las instrucciones de uso en el sitio web	Consultare le istruzioni per l'uso sul sito Web
	Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.	Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.	Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.	Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.	Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.
	Non-sterile	Non stérile	Unsteril	No estéril	Non sterile
	Authorized representative in the European Community/European Union	Représentant autorisé dans la Communauté européenne/l'Union européenne	Bevollmächtigter in der Europäischen Gemeinschaft/Europäischen Union	Representante autorizado en la Comunidad Europea/Unión Europea	Rappresentante autorizzato nella Comunità Europea/Unione Europea
	Manufacturer	Fabricant	Hersteller	Fabricante	Produttore
	Date of manufacture	Date de fabrication	Herstellungsdatum	Fecha de fabricación	Data di produzione
	Unique device identifier	Identifiant unique du dispositif	Einmalige Produktkennung	Identificador único del dispositivo	Identificatore univoco del dispositivo
	Importer	Importateur	Importeur	Importador	Importatore

	Nederlands	Dansk	Svenska	Ελληνικά	Português
	Modelnummer	Modelnummer	Modellnummer	Αριθμός μοντέλου	Número do modelo
	Catalogusnummer	Katalognummer	Katalognummer	Αριθμός καταλόγου	Número de catálogo

Symbol Legend ■ Légende des symboles ■ Zeichenerklärung ■ Significado de los símbolos ■ Legenda dei simboli ■ Lijst met symbolen ■ Symbolforklaring ■ Symbolförklaring ■ Υπόμνημα συμβόλων ■ Legenda de símbolos ■ Legenda se symboly ■ Jelmagyarázás ■ Legenda simboli ■ Vysvětlivky k symbolom ■ Symbolforklaring ■ Merkkien selitykset ■ Легенда на символите ■ Legendă de simboluri ■ Sümbolite seletus ■ Simbolių paaiškinimas ■ Simbolu skaidrojums ■ Sembol Açıklamaları ■ Условные обозначения ■ Legenda simbola

	Nederlands	Dansk	Svenska	Ελληνικά	Português
LOT	Lotnummer	Partinummer	Lotnummer	Αριθμός παρτίδας	Número de lote
QTY	Hoeveelheid	Mængde	Antal	Ποσότητα	Quantidade
	Inhoud	Indhold	Innehåll	Περιεχόμενο	Conteúdo
	Let op	Forsigtig	Var försiktig	Προσοχή	Aviso
	Raadpleeg de gebruiksaanwijzing of de elektronische gebruiksaanwijzing	Se brugsanvisningen, eller se elektronisk brugsanvisning	Se bruksanvisningen eller den elektroniska bruksanvisningen	Συμβουλευτείτε τις οδηγίες χρήσης ή συμβουλευτείτε τις ηλεκτρονικές οδηγίες χρήσης	Consultar as instruções de utilização ou consultar as instruções de utilização eletrônicas
	Niet gebruiken als de verpakking is beschadigd en raadpleeg de gebruiksaanwijzing	Må ikke anvendes, hvis emballagen er beskadiget, og se brugsanvisningen	Använd inte om förpackningen är skadad och se bruksanvisningen	Μη χρησιμοποιείτε εάν η συσκευασία έχει υποστεί ζημιά και συμβουλευτείτε τις οδηγίες χρήσης	Não utilizar se a embalagem estiver danificada e consultar as instruções de utilização
	Op een koele en droge plaats bewaren	Skal opbevares køligt og tørt	Förvara svalt och torrt.	Φυλάσσετε σε δροσερό και ξηρό χώρο	Guardar num local fresco e seco
MD	Medisch hulpmiddel	Medicinsk udstyr	Medicinsk utrustning	Ιατροτεχνολογικό προϊόν	Dispositivo médico
	Raadpleeg de gebruiksaanwijzing op de website	Se brugsanvisningen på webstedet	Se bruksanvisningen på webbplatsen	Συμβουλευτείτε τις οδηγίες χρήσης στον ιστότοπο	Consultar as instruções de utilização no site
Rx only	Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.	Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.	Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.	Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.	Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.
	Niet-steriel	Ikke-steril	Icke-steril	Μη αποστειρωμένο	Não esterilizado
EC REP	Geautoriseerde vertegenwoordiger in de Europese Gemeenschap/ Europese Unie	Autoriseret repræsentant i Det Europæiske Fællesskab/Den Europæiske Union	Auktoriserad representant inom Europeiska gemenskapen/ Europeiska unionen	Εξουσιοδοτημένος αντιπρόσωπος στην Ευρωπαϊκή Κοινότητα/ Ευρωπαϊκή Ένωση	Representante autorizado na Comunidade Europeia/União Europeia
	Fabrikant	Producent	Tillverkare	Κατασκευαστής	Fabricante
	Fabricagedatum	Fremstillingsdato	Tillverkningsdatum	Ημερομηνία κατασκευής	Data de fabrico
UDI	Unieke instrumentidentificatie	Unik udstyrsidentifikation	Unik produktidentifiering	Αποκλειστικό αναγνωριστικό ιατροτεχνολογικού προϊόντος	Identificador único de dispositivo
	Importeur	Importør	Importör	Εισαγωγέας	Importador

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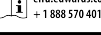
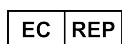
01/2024
10054690002 A
© Copyright 2024, Edwards Lifesciences LLC
All rights reserved.

Edwards Lifesciences LLC
One Edwards Way
Irvine, CA 92614 USA

Edwards Lifesciences B.V.
Verlengde Poolseweg 16
4818 CL Breda, Netherlands

Telephone +1.949.250.2500
+1.800.424.3278
FAX +1.949.250.2525

Web IFU

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Edwards Lifesciences GmbH