Edwards

Edwards Reusable Rail Assembly

Directory	
English (en).....	1
Français (fr).....	4
Deutsch (de).....	6
Español (es).....	8
Italiano (it).....	10
Nederlands (nl).....	12
Dansk (da).....	14
Svenska (sv).....	16
Ελληνικά (el).....	18
Português (pt).....	21
Česky (cs).....	23
Magyar (hu).....	25
Polski (pl).....	27
Slovensky (sk).....	29
Norsk (no).....	31
Suomi (fi).....	33
Български (bg).....	35
Română (ro).....	37
Eesti (et).....	39
Lietuvių (lt).....	41
Latviešu (lv).....	43
Türkçe (tr).....	45
Русский (ru).....	47
Srpski (sr).....	50
Hrvatski (hr).....	52
Figures ■ Figures ■ Abbildungen ■ Figuras ■ Figure ■ Afbeeldingen ■ Figurer ■ Figurer ■ Εικόνας ■ Figuras ■ Obrázky ■ Ábrák ■ Rysunki ■ Obrázky ■ Figurer ■ Kuvat ■ Фигури ■ Figuri ■ Joonised ■ Paveikslėliai ■ Attēli ■ Şekiller ■ Рисунки ■ Slike ■ Slike.....	54
Symbol Legend ■ Légende des symboles ■ Zeichenerklärung ■ Significado de los símbolos ■ Legenda dei simboli ■ Lijst met symbolen ■ Symbolforklaring ■ Symbolforklaring ■ Υπόμνημα συμβόλων ■ Legenda de símbolos ■ Legenda se symboly ■ Jelmagyarázat ■ Legenda symboli ■ Vysvetlivky k symbolom ■ Symbolforklaring ■ Merkkien selitykset ■ Легенда на символите ■ Legendă de simboluri ■ Sümbolite seletus ■ Simbolių paaiškinimas ■ Simbolu skaidrojums ■ Sembol Açıklamaları ■ Условные обозначения ■ Legenda sa simbolima ■ Legenda simbola.....	55

English

Edwards Reusable Rail Assembly

Accessory Care and Reprocessing Instructions for Model 10500RL

Rx only

1.0 Introduction

The Model 10500RL, in conjunction with an Edwards stabilizer, is intended to provide stability during use of an Edwards transcatheter system.

The Instructions For Use supplied with the respective Edwards transcatheter system provides information of the use and description of this accessory.

WARNING: Original packaging of the accessory is not intended for sterilization. This could result in compromised sterility.

WARNING: Failure to properly clean and sterilize accessory according to these instructions prior to use may result in infections, user injury, or patient injury.

The user is responsible for the qualification of any deviations from the recommended method of cleaning or sterilization.

2.0 How Supplied

The accessory is packaged separately, provided non-sterile and must be cleaned and sterilized according to this document before each use.

3.0 Cleaning Instructions

The accessory must be cleaned prior to each use in accordance with appropriate hospital procedures and the following recommended instructions and parameters. After cleaning, wrap the accessory in a standard sterilization wrap until disinfected or sterilized.

The accessory should be retired from further use and replaced if signs of cracking and/or gouges, corrosion, or other deterioration/damage are evident or knob does not fully translate in threads when rotated.

Avoid using metallic brushes, scrub pads, harsh chemicals (such as chlorine or caustic soda) or organic or ammoniated acids or solvents (such as acetone), or other abrasive cleaning aids when reprocessing the accessory as permanent damage to accessory may occur.

3.1 Instructions for Manual Cleaning:

Step	Procedure
1	Rinse the accessory under cold, running tap water for a minimum of 1 minute to remove gross soil. a) Use a soft-bristled brush for an additional minimum of 1 minute to aid in removal of soil, focusing on threads, crevices, and engraved areas. Ensure full depth of the feature is reached. b) While brushing, ensure that the knob remains fully retracted from the proximal clamp assembly threads and the internal knob threads are fully exposed (Figure 1).
2	Prepare a neutral pH enzymatic cleaning solution (such as Enzol or equivalent) per the manufacturer's recommendation.
3	Fully immerse the accessory in the cleaning solution and soak for a minimum of 1 minute or per manufacturer's recommendation, whichever is greater. When cleaning multiple accessories, avoid contact between accessories. a) Soak accessory such that knob threads are fully exposed (Figure 1).
4	After the soak time, use a soft-bristled brush and scrub the accessories for a minimum of 1 minute to remove any visible soil. a) Focus on threads, crevices, and engraved surfaces. b) While brushing, ensure that the knob remains fully retracted from the proximal clamp assembly (Figure 1). c) Scrub below water line.
5	While submerged wipe all surfaces of the accessory for a minimum of 1 minute with a clean cloth (such as a lint free cloth) moistened with prepared detergent solution.
6	Remove the accessory from the cleaning solution and rinse with cold, running tap water for a minimum of 5 minutes. a) Ensure that running water passes through sliding surfaces and threads.
7	Dry accessory with a clean lint free cloth.
8	If soil is visible, repeat cleaning process.
9	After cleaning, the accessory should be wrapped in a standard sterilization wrap until disinfected or sterilized.

3.2 Instructions for Automated Cleaning

Step	Procedure																				
1	Rinse the accessory under cold, running tap water for a minimum of 1 minute to remove gross soil. a) Use a soft-bristled brush for an additional minimum of 1 minute to aid in removal of soil, focusing on threads, crevices, and engraved areas. Ensure full depth of the feature is reached. b) While brushing, ensure that the knob remains fully retracted from the proximal clamp assembly threads and the internal knob threads are fully exposed (Figure 1).																				
2	Load accessory into the automated washer such that knob threads remain fully exposed.																				
3	Set motor speed to HIGH, if applicable, and select the cycle and cycle parameters per the following recommendations: <table><tr><th>PHASE</th><th>RECIRCULATION TIME (MINUTES)</th><th>TEMPERATURE</th><th>DETERGENT TYPE AND CONCENTRATION</th></tr><tr><td>Pre-wash</td><td>2:00</td><td>Cold tap water</td><td>N/A</td></tr><tr><td>Wash</td><td>3:00¹</td><td>Hot utility water</td><td>Alkaline detergent (LF2100 or equivalent) per the detergent manufacturer's recommendation</td></tr><tr><td>Rinse</td><td>2:00</td><td>60 °C utility water (set point)</td><td>N/A</td></tr><tr><td>Drying</td><td>8:00</td><td>90 °C (194 °F)</td><td>N/A</td></tr></table> ¹Or per automated cleaner manufacturer's recommendation, whichever is greater	PHASE	RECIRCULATION TIME (MINUTES)	TEMPERATURE	DETERGENT TYPE AND CONCENTRATION	Pre-wash	2:00	Cold tap water	N/A	Wash	3:00 ¹	Hot utility water	Alkaline detergent (LF2100 or equivalent) per the detergent manufacturer's recommendation	Rinse	2:00	60 °C utility water (set point)	N/A	Drying	8:00	90 °C (194 °F)	N/A
PHASE	RECIRCULATION TIME (MINUTES)	TEMPERATURE	DETERGENT TYPE AND CONCENTRATION																		
Pre-wash	2:00	Cold tap water	N/A																		
Wash	3:00 ¹	Hot utility water	Alkaline detergent (LF2100 or equivalent) per the detergent manufacturer's recommendation																		
Rinse	2:00	60 °C utility water (set point)	N/A																		
Drying	8:00	90 °C (194 °F)	N/A																		
4	When unloading, check moving parts and surrounding surfaces for complete removal of visible soil. If necessary, repeat automated cleaning cycle or use manual cleaning.																				
5	After cleaning, the accessory should be wrapped in a standard sterilization wrap until disinfected or sterilized.																				

4.0 Disinfection (Optional*)

4.1 Instructions for Automated Thermal Disinfection

Step	Procedure			
1	Load accessory into washer-disinfector such that knob threads are fully exposed (Figure 1).			
2	Set the motor speed to HIGH, if applicable, and select the following recommended parameters:			
	PHASE	RECIRCULATION TIME (MINUTES)	WATER TEMPERATURE	DETERGENT TYPE AND CONCENTRATION
	Thermal rinse	5:00	90 °C (194 °F)	N/A
3	After cleaning, the accessory should be wrapped in a standard sterilization wrap until disinfected or sterilized.			

*Automated thermal disinfection is recommended per facility guidelines as an added safety factor for personnel.

5.0 Sterilization Instructions

The accessory must be sterilized prior to each use in accordance with appropriate hospital procedures and the following recommended instructions and parameters. Institution should use procedures that include biological indicator to determine the effectiveness of the sterilization.

The accessory should be retired from further use and replaced if signs of cracking and/or gouges, corrosion, or other deterioration/damage are evident or knob does not fully translate in threads when rotated.

When sterilizing multiple accessories in one cycle, ensure accessories or trays are not stacked and sterilizer max load is not exceeded.

Sterilizer Type	Configuration	Minimum Temperature	Minimum Exposure Time	Minimum Dry Time
Gravity Displacement	Double Wrapped ³	121 °C (250 °F)	30 minutes	55 minutes
Pre-vacuum	Double Wrapped ³	132 °C (270 °F)	4 minutes	75 minutes
Pre-vacuum	Double Wrapped ³	134 °C (274 °F)	3 minutes	75 minutes

³ Halyard Health H300 or equivalent loaded in configuration depicted in Figure 1

6.0 Storage

Store in double wrapped configuration and/or in a cool and dry place.

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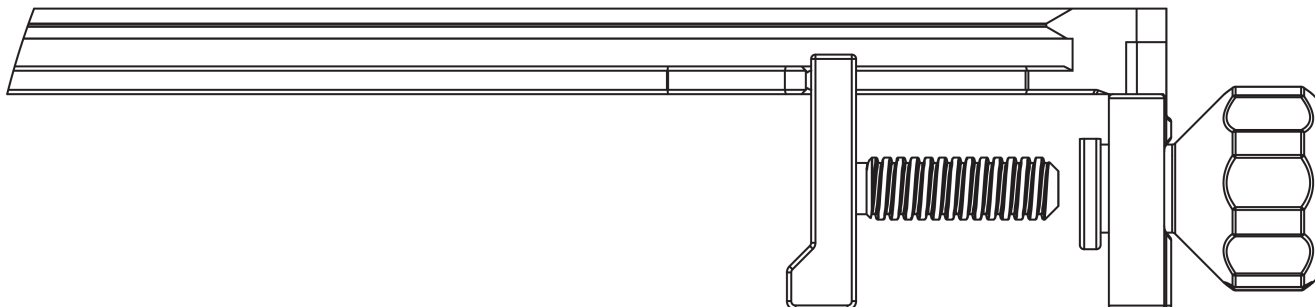
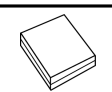
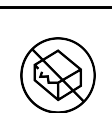
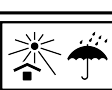
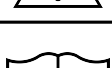
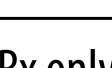
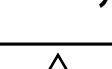
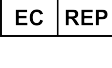
Figure 1: Proximal Clamp Assembly and Internal Knob Threads Fully Exposed

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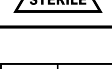
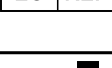
	English	Français	Deutsch	Español	Italiano
	Model Number	Référence	Modellnummer	Número de modelo	Numero di modello
	Catalogue Number	Référence catalogue	Artikelnummer	Número de catálogo	Numero di catalogo
	Lot Number	N° du lot	Chargenbezeichnung	Número de lote	Numero di lotto
	Quantity	Quantité	Menge	Cantidad	Quantità
	Contents	Contenu	Inhalt	Contenido	Contenuti
	Caution	Avertissement	Vorsicht	Aviso	Attenzione
	Consult instructions for use or consult electronic instructions for use	Consultez le mode d'emploi sur papier ou le mode d'emploi au format électronique	Gebrauchsanweisung beachten oder elektronische Gebrauchsanweisung beachten	Consulte las instrucciones de uso o las instrucciones de uso electrónicas	Consultare le istruzioni per l'uso o le istruzioni per l'uso elettroniche
	Do not use if package is damaged and consult instructions for use	Ne pas utiliser si le conditionnement est endommagé et consulter le mode d'emploi	Bei beschädigter Verpackung nicht verwenden und Gebrauchsanweisung beachten	No lo utilice si el envase está dañado y consulte las instrucciones de uso	Non utilizzare se la confezione è danneggiata e consultare le istruzioni per l'uso
	Store in a cool, dry place	Conserver dans un endroit frais et sec	Das Produkt an einem kühlen, trockenen Ort lagern	Guárdese en un lugar fresco y seco	Conservare in un luogo fresco e asciutto
	Medical device	Dispositif médical	Medizinprodukt	Producto sanitario	Dispositivo medico
	Consult instructions for use on the website	Consulter le mode d'emploi sur le site Web	Gebrauchsanweisung auf der Website beachten	Consulte las instrucciones de uso en el sitio web	Consultare le istruzioni per l'uso sul sito Web
	Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.	Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.	Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.	Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.	Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.
	Non-sterile	Non stérile	Unsteril	No estéril	Non sterile
	Authorized representative in the European Community/European Union	Représentant autorisé dans la Communauté européenne/l'Union européenne	Bevollmächtigter in der Europäischen Gemeinschaft/Europäischen Union	Representante autorizado en la Comunidad Europea/Unión Europea	Rappresentante autorizzato nella Comunità Europea/Unione Europea
	Manufacturer	Fabricant	Hersteller	Fabricante	Produttore
	Date of manufacture	Date de fabrication	Herstellungsdatum	Fecha de fabricación	Data di produzione
	Unique device identifier	Identifiant unique du dispositif	Einmalige Produktkennung	Identificador único del dispositivo	Identificatore univoco del dispositivo
	Importer	Importateur	Importeur	Importador	Importatore

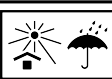
Symbol Legend ■ Légende des symboles ■ Zeichenerklärung ■ Significado de los símbolos ■ Legenda dei simboli ■ Lijst met symbolen
 ■ Symbolforklaring ■ Symbolförklaring ■ Υπόμνημα συμβόλων ■ Legenda de símbolos ■ Legenda se symboly ■ Jelmagyarázat ■ Legenda symboli
 ■ Vysvetlivky k symbolom ■ Symbolforklaring ■ Merkkien selitykset ■ Легенда на символите ■ Legendă de simboluri ■ Sümbolite seletus
 ■ Simbolių paaiškinimas ■ Simbolu skaidrojums ■ Sembol Açıklamaları ■ Условные обозначения ■ Legenda sa simbolima ■ Legenda simbola

	Nederlands	Dansk	Svenska	Ελληνικά	Português
	Modelnummer	Modelnummer	Modellnummer	Αριθμός μοντέλου	Número do modelo
	Catalogusnummer	Katalognummer	Katalognummer	Αριθμός καταλόγου	Número de catálogo
	Lotnummer	Partinummer	Lotnummer	Αριθμός παρτίδας	Número de lote
	Hoeveelheid	Mængde	Antal	Ποσότητα	Quantidade
	Inhoud	Indhold	Innehåll	Περιεχόμενο	Conteúdo
	Let op	Forsigtig	Var försiktig	Προσοχή	Aviso
	Raadpleeg de gebruiksaanwijzing of de elektronische gebruiksaanwijzing	Se brugsanvisningen, eller se elektronisk brugsanvisning	Se bruksanvisningen eller den elektroniska bruksanvisningen	Συμβουλευτείτε τις οδηγίες χρήσης ή συμβουλευτείτε τις ηλεκτρονικές οδηγίες χρήσης	Consultar as instruções de utilização ou consultar as instruções de utilização eletrônicas
	Niet gebruiken als de verpakking is beschadigd en raadpleeg de gebruiksaanwijzing	Må ikke anvendes, hvis emballagen er beskadiget, og se brugsanvisningen	Använd inte om förpackningen är skadad och se bruksanvisningen	Μη χρησιμοποιείτε εάν η συσκευασία έχει υποστεί ζημιά και συμβουλευτείτε τις οδηγίες χρήσης	Não utilizar se a embalagem estiver danificada e consultar as instruções de utilização
	Op een koele en droge plaats bewaren	Skal opbevares køligt og tørt	Förvara svalt och torrt.	Φυλάσσετε σε δροσερό και ξηρό χώρο	Guardar num local fresco e seco
	Medisch hulpmiddel	Medicinsk udstyr	Medicinteknisk produkt	Ιατροτεχνολογικό προϊόν	Dispositivo médico
	Raadpleeg de gebruiksaanwijzing op de website	Se brugsanvisningen på webstedet	Se bruksanvisningen på webbplatsen	Συμβουλευτείτε τις οδηγίες χρήσης στον ιστότοπο	Consultar as instruções de utilização no site
	Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.	Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.	Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.	Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.	Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.
	Niet-steriel	Ikke-steril	Icke-sterilt	Μη αποστειρωμένο	Não esterilizado
	Geautoriseerde vertegenwoordiger in de Europese Gemeenschap/ Europese Unie	Autoriseret repræsentant i Det Europæiske Fællesskab/Den Europæiske Union	Auktoriserad representant inom Europeiska gemenskapen/Europeiska unionen	Εξουσιοδοτημένος αντιπρόσωπος στην Ευρωπαϊκή Κοινότητα/ Ευρωπαϊκή Ένωση	Representante autorizado na Comunidade Europeia/ União Europeia
	Fabrikant	Producent	Tillverkare	Κατασκευαστής	Fabricante
	Fabricagedatum	Fremstillingsdato	Tillverkningsdatum	Ημερομηνία κατασκευής	Data de fabrico
	Unieke instrumentidentificatie	Unik udstyrsidentifikation	Unik produktidentifisering	Αποκλειστικό αναγνωριστικό ιατροτεχνολογικού προϊόντος	Identificador único de dispositivo
	Importeur	Importør	Importör	Εισαγωγέας	Importador

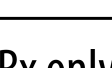
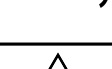
Symbol Legend ■ Légende des symboles ■ Zeichenerklärung ■ Significado de los símbolos ■ Legenda dei simboli ■ Lijst met symbolen
 ■ Symbolforklaring ■ Symbolförklaring ■ Υπόμνημα συμβόλων ■ Legenda de símbolos ■ Legenda se symboly ■ Jelmagyarázat ■ Legenda simboli
 ■ Vysvetlivky k symbolom ■ Symbolforklaring ■ Merkkien selitykset ■ Легенда на символите ■ Legendă de simboluri ■ Sümbole seletus
 ■ Simbolių paaiškinimas ■ Simbolu skaidrojums ■ Sembol Açıklamaları ■ Условные обозначения ■ Legenda sa simbolima ■ Legenda simbola

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	Suomi	Български	Română	Eesti	Lietuvių
#	Mallinúmero	Номер на модела	Număr model	Mudeli number	Modelio numeris
REF	Luettelonúmero	Каталожен номер	Număr de catalog	Kataloogi number	Katalogo numeris
LOT	Eränumero	Партиден номер	Număr de lot	Partii number	Partijos numeris
QTY	Määrä	Количество	Cantitate	Kogus	Kiekis
	Sisältö	Съдържание	Cuprins	Sisu	Turiny
	Tärkeä huomautus	Внимание	Atenție	Ettevaatust	Perspėjimas
	Katso käyttöohjeet tai sähköiset käyttöohjeet	Направете справка с инструкциите за употреба или електронните инструкции за употреба	Consultați instrucțiunile de utilizare sau consultați instrucțiunile de utilizare în format electronic	Tutvuge paberkandjal või elektroonilise kasutusjuhisega	Žr. naudojimo instrukcijas arba elektronines naudojimo instrukcijas
	Älä käytä, jos pakkaus on vahingoittunut, ja katso käyttöohjeet	Да не се използва, ако опаковката е повредена, и направете справка с инструкциите за употреба	A nu se utiliza dacă ambalajul este deteriorat. Consultați instrucțiunile de utilizare	Ärge kasutage, kui pakend on kahjustatud, ja lugege kasutusjuhiseid	Nenaudoti, jei pakuotė pažeista, ir žr. naudojimo instrukcijas
	Säilytettävä kuivassa ja viileässä	Да се съхранява на хладно, сухо място	Stocați într-un loc rece și uscat	Säilitage jahedas ja kuivas kohas.	Laikyti vėsioje, sausoje vietoje
MD	Lääkinnällinen laite	Медицинско изделие	Dispozitiv medical	Meditsiiniseade	Medicinos priemonė
	Katso käyttöohjeet verkkosivustolta	Направете справка с инструкциите за употреба в уебсайта	Consultați instrucțiunile de utilizare de pe site-ul web	Palun lugege kasutusjuhiseid veebisaidilt	Žr. naudojimo instrukcijas interneto svetainėje
Rx only	Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.	Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.	Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.	Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.	Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.
	Epästeriili	Нестерилно	Nesteril	Mittesteriilne	Nesterilus
EC REP	Valtuutettu edustaja Euroopan yhteisössä / Euroopan unionissa	Упълномощен представител в Европейската общност/ Европейския съюз	Reprezentant autorizat în Comunitatea Europeană/ Uniunea Europeană	Volitatud esindaja Euroopa Ühenduses / Euroopa Liidus	Igaliotasis atstovas Europos Bendrijoje / Europos Sąjungoje
	Valmistaja	Производител	Producător	Tootja	Gamintojas
	Valmistuspäivämäärä	Дата на производство	Data fabricației	Tootmiskuupäev	Pagaminimo data
UDI	Laitteen yksilöivä tunnus	Уникален идентификатор на изделието	Identificator unic al dispozitivului	Seadme kordumatu identifitseerimistunnus	Unikalasis priemonės identifikatorius
	Maahantuoja	Вносител	Importator	Importija	Importuotojas

Symbol Legend ■ Légende des symboles ■ Zeichenerklärung ■ Significado de los símbolos ■ Legenda dei simboli ■ Lijst met symbolen
 ■ Symbolforklaring ■ Symbolförklaring ■ Υπόμνημα συμβόλων ■ Legenda de símbolos ■ Legenda se symboly ■ Jelmagyarázat ■ Legenda simboli
 ■ Vysvetlivky k symbolom ■ Symbolforklaring ■ Merkkien selitykset ■ Легенда на символите ■ Legendă de simboluri ■ Sümbolite seletus
 ■ Simbolių paaiškinimas ■ Simbolu skaidrojums ■ Sembol Açıklamaları ■ Условные обозначения ■ Legenda sa simbolima ■ Legenda simbola

	Latviešu	Türkçe	Русский	Srpski	Hrvatski
	Modeļa numurs	Model Numarası	Номер модели	Broj modela	Broj modela
	Kataloga numurs	Katalog Numarası	Номер по каталогу	Kataloški broj	Kataloški broj
	Partijas numurs	Lot Numarası	Номер партии	Broj partije	Broj serije
	Daudzums	Miktar	Количество	Količina	Količina
	Saturs	İçindekiler	Содержимое	Sadržaj	Sadržaj
	Uzmanību!	Dikkat	Предостережение	Oprez	Oprez
	Skatiet lietošanas instrukcijas vai elektroniskās lietošanas instrukcijas	Kullanım talimatlarına veya elektronik kullanım talimatlarına başvurun	См. инструкции по применению либо электронные инструкции по применению	Pogledajte uputstvo za upotrebu ili elektronsko uputstvo za upotrebu	Pogledajte upute za upotrebu ili elektroničke upute za upotrebu
	Nelietot, ja iepakojums ir bojāts, un skatīt lietošanas instrukciju	Ambalaj hasarlıysa kullanmayın ve kullanım talimatlarına başvurun	Не использовать, если упаковка повреждена; использовать в соответствии с инструкцией	Ne koristite ako je pakovanje oštećeno i pogledajte uputstvo za upotrebu	Nemojte upotrebljavati ako je pakiranje oštećeno i pogledajte upute za upotrebu
	Uzglabāt vēsā, sausā vietā	Serin, kuru yerde saklayın	Хранить в прохладном, сухом месте	Čuvati na hladnom i suvom mestu	Pohranite na hladnom, suhom mjestu
	Medicīniska ierīce	Tıbbi cihaz	Медицинское устройство	Medicinsko sredstvo	Medicinski uređaj
	Skatīt lietošanas instrukciju tīmekļa vietnē	Web sitesindeki kullanım talimatlarına başvurun	См. инструкции по применению на веб-сайте	Pogledajte uputstvo za upotrebu na internet stranici	Pogledajte upute za upotrebu na internetskoj stranici
	Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.	Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.	Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.	Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.	Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.
	Nesterils	Steril Değildir	Нестерильно	Nesterilno	Nesterilno
	Pilnvarotais pārstāvis Eiropas Kopienā/Eiropas Savienībā	Avrupa Birliği'nde/Avrupa Topluluğu'nda Yetkili Temsilci	Официальный представитель в Европейском сообществе/Европейском союзе	Ovlašćeni predstavnik u Evropskoj zajednici / Evropskoj Uniji	Ovlašteni predstavnik u Europskoj zajednici / Europskoj uniji
	Ražotājs	Üretici	Производитель	Proizvođač	Proizvođač
	Izgatavošanas datums	Üretim tarihi	Дата производства	Datum proizvodnje	Datum proizvodnje

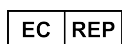
Symbol Legend ■ Légende des symboles ■ Zeichenerklärung ■ Significado de los símbolos ■ Legenda dei simboli ■ Lijst met symbolen ■ Symbolforklaring ■ Symbolförklaring ■ Υπόμνημα συμβόλων ■ Legenda de símbolos ■ Legenda se symboly ■ Jelmagyarázat ■ Legenda symboli ■ Vysvetlivky k symbolom ■ Symbolforklaring ■ Merkkien selitykset ■ Легенда на символите ■ Legendă de simboluri ■ Sümbole seletus ■ Simbolių paaiškinimas ■ Simbolu skaidrojums ■ Sembol Açıklamaları ■ Условные обозначения ■ Legenda sa simbolima ■ Legenda simbola

	Latviešu	Türkçe	Русский	Srpski	Hrvatski
	Unikāls ierīces identifikators	Benzersiz cihaz tanımlayıcısı	Уникальный идентификатор устройства	Jedinstveni identifikator sredstva	Jedinstveni identifikator proizvoda
	Importētājs	İthalatçı	Импортер	Uvoznik	Uvoznik

Note: The labeling of this product may not contain every symbol depicted in this legend. ■ **Remarque :** il est possible que certains symboles de cette légende n'apparaissent pas sur les étiquettes de ce produit. ■ **Hinweis:** Die Kennzeichnung dieses Produkts enthält möglicherweise nicht jedes Symbol, das in dieser Zeichenerklärung abgebildet ist. ■ **Nota:** Es posible que no todos los símbolos mostrados en esta leyenda aparezcan en el etiquetado de este producto. ■ **Nota:** l'etichetta di questo prodotto potrebbe non contenere tutti i simboli riportati in questa legenda. ■ **Opmerking:** De etikettering van dit product bevat mogelijk niet elk symbool dat in deze legenda wordt weergegeven. ■ **Bemærk:** Produktets mærkning omfatter muligvis ikke alle symboler gengivet i denne forklaring. ■ **Obs!** Märkningen av produkten innehåller eventuellt inte alla symboler som avbildas i denna symbolförklaring. ■ **Σημείωση:** Η σήμανση του προϊόντος ενδέχεται να μην περιέχει κάθε σύμβολο που αναφέρεται στο παρόν υπόμνημα. ■ **Nota:** a rotulagem deste produto pode não conter todos os símbolos ilustrados nesta legenda. ■ **Poznámka:** Označení tohoto výrobku nemusí obsahovat všechny symboly uvedené v této legendě. ■ **Megjegyzés:** A termék címkéje nem feltétlenül tartalmazza az ezen jelmagyarázattal szereplő összes szimbólumot. ■ **Uwaga:** Etykieta tego wyrobu może nie zawierać wszystkich symboli przedstawionych w niniejszej legendzie. ■ **Poznámka:** Na štítku tohto produktu sa nemusia nachádzať všetky symboly uvedené v týchto vysvetlivkách. ■ **Merk:** Merkingen av dette produktet inneholder kanskje ikke alle symbolene som er gjengitt i denne symbolforklaringen. ■ **Huomautus:** Tämän tuotteen merkinnät eivät välttämättä sisällä kaikkia tässä selityksessä olevia symboleita. ■ **Забележка:** Етикетите по този продукт може да не съдържат всеки символ, представен в тази легенда. ■ **Notă:** este posibil ca eticheta acestui produs să nu conțină fiecare simbol descris în această legendă. ■ **Märkus.** Toote esinev märgistus ei pruugi sisaldada kõiki tingmärkide seletuses välja toodud sümboleid. ■ **Pastaba.** Šio gaminio etiketėje gali nebūti visų šiame paaiškinime pavaizduotų simbolių. ■ **Piezīme.** Ši produkta marķējuma var nebūt visu simbolu, kuri iekļauti šajā simbolu skaidrojumā. ■ **Not:** Bu ürünün etiketi bu açıklamada verilen her sembolü içermeyebilir. ■ **Примечание.** Маркировка изделия может содержать не все условные обозначения. ■ **Napomena:** Na ambalaži proizvoda se možda ne nalaze svi simboli. ■ **Napomena:** oznaka ovoga proizvoda možda ne sadrži svaki simbol prikazan u ovoj legendi simbola.

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08/2024
10060807001 A
© Copyright 2024, Edwards Lifesciences LLC
All rights reserved.

Edwards Lifesciences LLC
One Edwards Way
Irvine, CA 92614 USA

Edwards Lifesciences B.V.
Verlengde Poolseweg 16
4818 CL Breda, Netherlands

Telephone +1.949.250.2500
+1.800.424.3278
FAX +1.949.250.2525

Web IFU

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Edwards Lifesciences GmbH